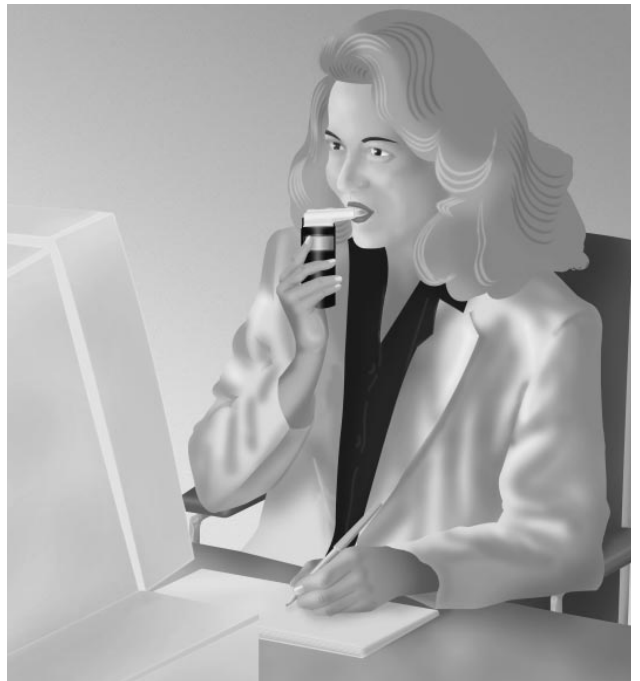
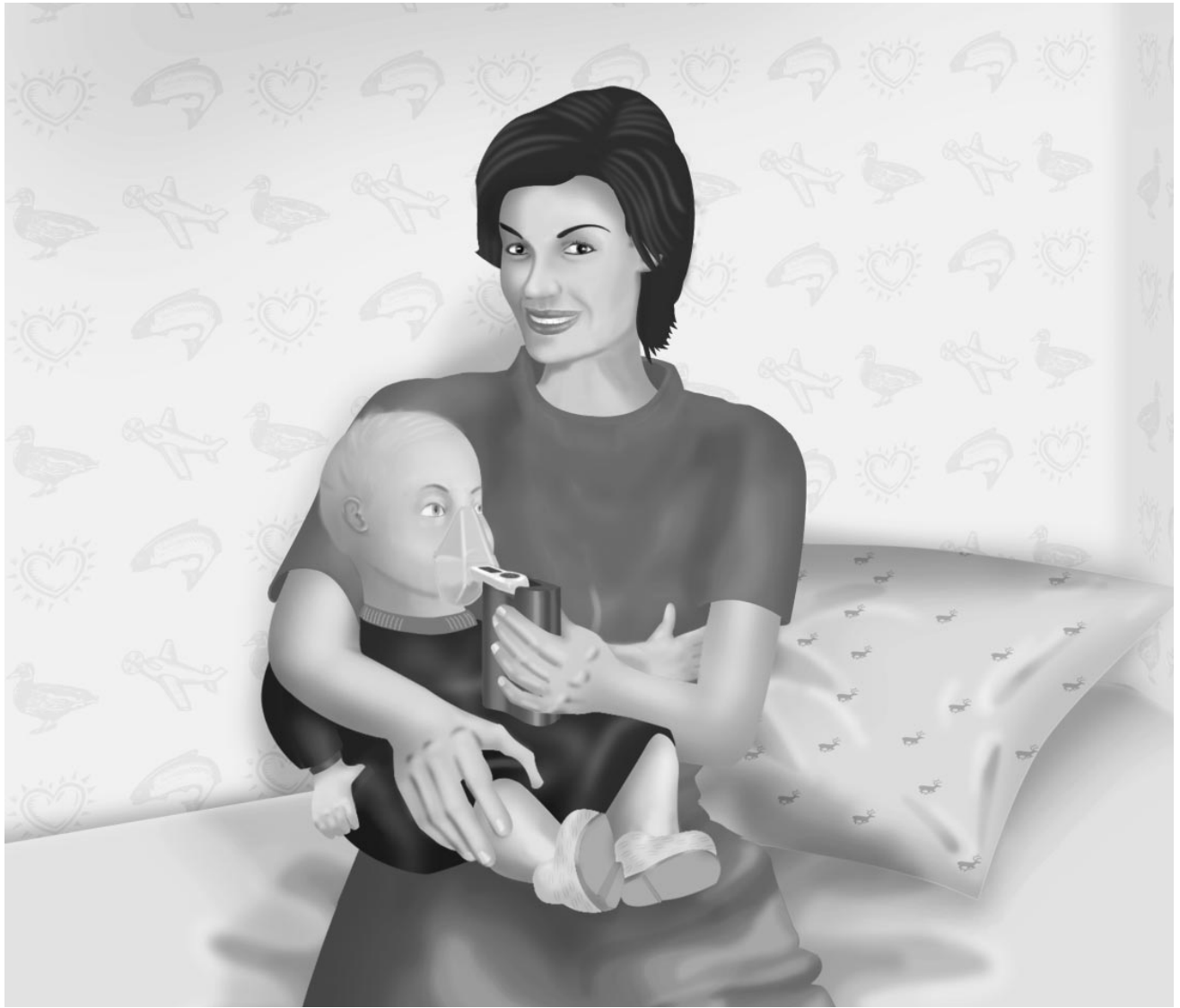
# **COMP** *Air*<sup>®</sup> *Elite*

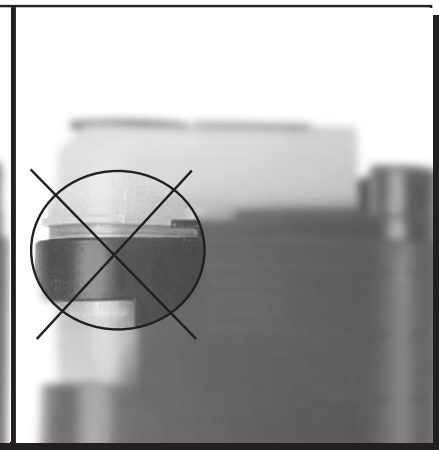
Compressor Nebulizer System

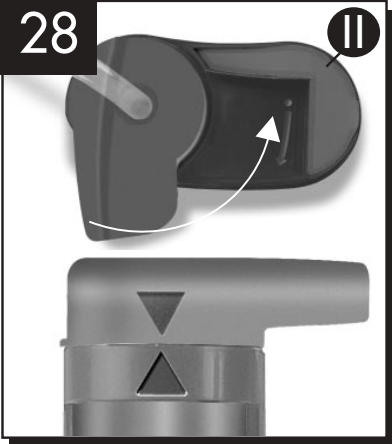
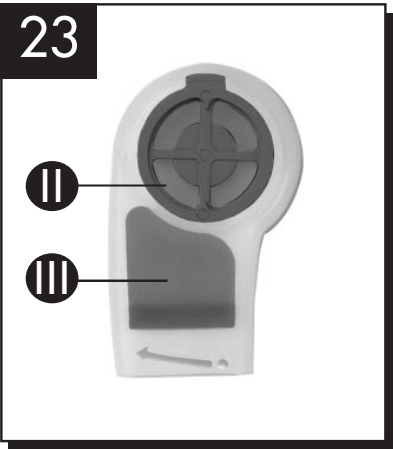
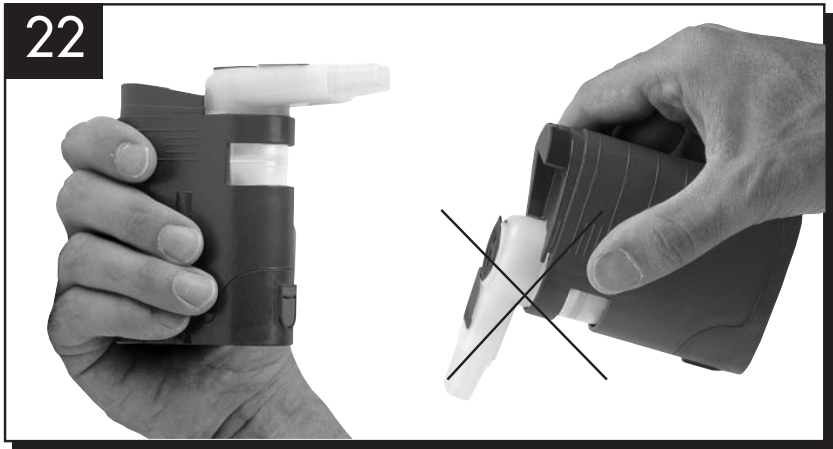
NE-C21

U  
Y  
te  
no  
It  
an  
yo  
ch  
  
C  
or  
  
In  
K  
A  
C  
In  
C  
T  
S  
V  
  
2

# TABLE OF CONTENTS

---

Your CompAir® *Elite* offers you the best in compressor nebulizer technology. The Elite has been a clinically proven compressor nebulizer system for delivering prescription drugs to the lungs. It is also the most convenient for you because of the compact size and the shorter treatment time. For even greater convenience ask your supplier about the optional battery pack. Thank you for choosing the CompAir Elite.

**CAUTION:** Federal law restricts this device to sale by or on the order of a physician.

Important Safeguards . . . . .	3,4,5
Know Your Unit. . . . .	5
Accessories. . . . .	6
Operating Instructions . . . . .	7,8
Instructions for Accessories . . . . .	9
Cleaning and Maintenance. . . . .	10,11
Troubleshooting . . . . .	12,13
Specifications. . . . .	14,15
Warranty . . . . .	16

# IMPORTANT SAFEGUARDS

---

Rx only

**⚠ IMPORTANT: Read instruction manual carefully before using this device.**

## PRECAUTIONS

- A. Use this device only as an aerosol therapy system, following the directions of your doctor. Any other kind of use constitutes an improper use and is therefore dangerous. The manufacturer cannot be held liable for any damage caused by improper, incorrect or unreasonable use, or if the unit is connected to electrical installations which do not conform to safety regulations.
- B. Keep this manual for future reference and for technical information.
- C. Do not operate the unit in the presence of anesthetic gases which might be flammable when they come in contact with air, oxygen or nitrous oxide.
- D. The correct device functioning can be affected by electromagnetic interferences exceeding the limits indicated by the European standard EN60601-1-2 and FCC 47CFR part 15.
- E. In case of failure and/or malfunction, turn the device off and read the instruction manual. Should the device keep on operating improperly, even if the instructions for use have been followed, contact Omron customer service. **Never open the compressor main unit.**
- F. For any repairs, refer only to a qualified technical service representative authorized by the manufacturer. Otherwise the safety of the device can be compromised.
- G. When using any electrical appliance certain important safety measures must always be observed, in particular;
  - use only manufacturer's accessories and components;

## IMPORTANT SAFEGUARDS

---

- use only manufacturer's accessories and components;
- do not immerse the device in water;
- do not touch the device with wet or moist hands;
- do not leave the device exposed to weather;
- the device, when operating, must be placed on a stable and horizontal surface;
- when used by children or persons with disabilities this unit requires adult supervision;
- do not pull on the power supply cord or the device to unplug the unit from the electrical outlet.

- H. For travel outside North America, you may need a plug adapter for the power supply. Before plugging the device into the power supply, make sure that the electric data, indicated on the rating label on the bottom of the unit and/or of the AC adapter, correspond to the power supply ones. If the plug the device is equipped with is not compatible with the power supply, refer to qualified personnel for replacing the plug with a suitable one. Generally, it is not advisable to use any adapter, simple or multiple, and/or extension cord. In case their use is necessary, make sure they are in compliance with the safety standards, paying attention they do not exceed the maximum values indicated on the adapters and extension cords.
- I. Do not leave the unit plugged in when not in use: unplug the device from the electrical supply when not operating.
- J. Follow instructions carefully for operating the device. The manufacturer cannot be held responsible for any damage caused by incorrect operation.
- K. In case of failure or damage to the adapter only use manufacturer's original replacement parts. The use of an improper adapter might modify the performances without assuring the safety conditions of the unit.

## IMPORTANT SAFEGUARDS

- L. The adapter cords should always be fully unwound in order to prevent dangerous overheating.
- M. Before performing any maintenance or cleaning operation, disconnect the device from the electrical supply.
- N. Some parts of the unit are so small that children could swallow them; therefore keep the device out of the reach of children.
- O. Remember to:
  - use this device only with the medications prescribed by your doctor;
  - follow the directions of your doctor when taking your treatments.

## KNOW YOUR UNIT

1. Compressor main unit
2. Air filter holder
3. Nebulizer kit
4. Inhalation valve
5. Exhalation valve
6. Mouthpiece
7. Nebulizer chamber
8. Indicator light
9. On/Off button
10. AC Adapter Jack
11. Optional Battery Pack Jack

## ACCESSORIES

---

### **Included:**

Nebulizer with inhalation/exhalation (respiratory) valves and a micronization spare cone

Unit lid

Mouthpiece – 2 pcs.

Air filters – 5 pcs.

Carrying bag

Universal AC adapter

### **Optional:**

Adult and pediatric masks

DC Kit (includes rechargeable battery pack and Car lighter adapter)

Car cigarette lighter adapter

Rechargeable battery pack

# OPERATING INSTRUCTIONS

---

 Check the unit for proper operation before each use.

**This unit has been designed for intermittent use. Normal treatment time should be 5 to 10 minutes. Turn off the unit after 20 minutes of use, and allow unit to cool down 40 minutes before turning it on again.**

1. As soon as you have unpacked the device check it for visible damages or defects.
2. Check that accessories are complete and not damaged.
3. Before using the device, proceed with the cleaning operations as described in the “CLEANING AND MAINTENANCE” section.
4. Remove the lid from the main unit body (pic. 11).
5. Make sure that the nebulizer top is in the “Stand-by” position (pic. 12), i.e.:
  - the nebulizer top is seated snugly on the nebulizer body and it is properly positioned in the nebulizer seat;
  - the arrow indicators between the nebulizer top and the nebulizer seat are aligned.
6. Pull the nebulizer kit out from the “stand-by” position (pic. 13).
7. Open the nebulizer by pulling the top up (pic. 13).
8. Make sure that the micronization cone (blue color) is correctly positioned inside the nebulizer chamber (pic. 15).
9. Pour the prescribed medication into the nebulizer body (pic. 18) as instructed by your doctor, paying attention not to exceed the maximum allowed quantity (7 ml.).

**Caution: Never pour medication into the nebulizer seat (pump) area of the compressor. Pump damage will occur. Place medication in the nebulizer cup only.**

# OPERATING INSTRUCTIONS

---

10. Close the nebulizer by firmly pressing the top onto the nebulizer body until a click is heard (pic. 17).
11. Place the nebulizer kit in the “stand-by” position, making sure it is firmly pressed into its seat (pic. 18).
12. Rotate the nebulizer top until the desired position is reached (pic. 19). Connect the mouthpiece or mask (not included).
13. Connect the power supply cord to the adapter and insert the plug into the electrical outlet. Make sure that the electric ratings shown on the adapter rating label correspond to the electrical supply. Insert the AC plug of the adapter into the socket on the side of the main unit (pic. 20).
14. The unit can be alternatively used with DC Auto Cable plug (not included) (pic. 9) or with the rechargeable battery pack (not included) (pic. 10).
15. Turn the device on by pushing the ON/OFF button (pic. 21) and inhale as prescribed by your doctor.
16. For correct operation of the device, do not position the unit more than 30°. Positioning the unit beyond 90° will result in medication spillage (pic. 22).
17. When treatment has been completed, turn the unit off. Unplug the adapter, remove the mouthpiece or mask, rotate the nebulizer top into the “STAND-BY” position, pull out the nebulizer kit and proceed with cleaning operations as described in the “CLEANING AND MAINTENANCE” section.

# INSTRUCTIONS FOR ACCESSORIES

## HOW TO USE THE MOUTHPIECE AND THE MASK

- A. Make sure inhalation valve (II) and exhalation valve (III) are correctly positioned and seated on the nebulizer kit (pic. 23).
- B. Insert the mouthpiece or the mask into the nebulizer top (pic. 24).
- C. Close your lips around the mouthpiece or place the mask onto your face and start treatment as recommended by your doctor.
- D. Once treatment is finished and cleaning operations have been carried out, you may store the mouthpiece inside the nebulizer top (pic. 25).

## CLEANING AND MAINTENANCE

### CLEANING AND MAINTENANCE OF THE ELITE COMPRESSOR UNIT

- A. Wipe exterior surfaces with a moist cloth and dry the unit immediately after cleaning operation has been carried out.

#### **ATTENTION**

- **Do not get the unit wet or submerge the unit into liquids.**
- **Do not use abrasive substances to clean the unit.**

- B. The filter must be replaced after one month of use, or when it turns grey, or after the device has been in storage for 3 months or longer (pic. 30).

#### **ATTENTION**

- **Use only manufacturer's original air filters.**
- **Do not operate the unit without an air filter.**

## CLEANING AND MAINTENANCE

---

**WARNING:** If fluid gets into the pump area, wipe unit off with dry cloth and let the unit sit for 5 minutes while in an upside down position. After 5 minutes run pump for 2 minutes without nebulizer kit installed.

### CLEANING AND MAINTENANCE OF THE NEBULIZER KIT

- A. Open the nebulizer, remove the micronization cone (blue color) from the chamber (pic. 31).
- B. Boil the chamber and the top and the cone in water (max. 5 min.) or autoclave them (max. 121°C, max. 10 min).
- C. Drain the components, assemble and insert into the unit.
- D. Turn on the device and let it run freely for about 5 minutes.

This operation will dry the components and will prevent accumulation of residual medication which could obstruct the air/medication passage.

### ATTENTION

- **The nebulizer kit should be replaced approximately after 6 months of use, or if the nebulizer kit does not operate properly.**
- **Only use manufacturer's original replacement nebulizer kits.**

### CLEANING AND MAINTENANCE OF MOUTHPIECE

- A. Boil in water (max. 5 minutes) or autoclave (max. 121°C, max. 10 min.).

# CLEANING AND MAINTENANCE

B. Drain the accessories and let them air dry.

## CLEANING AND MAINTENANCE OF MASKS

1. Disinfect the masks using cold disinfecting liquids, e.g. bleach and water, vinegar
2. Rinse carefully and thoroughly under running warm water and dry.

**⚠ ATTENTION: do not boil or autoclave the masks.**

# TROUBLESHOOTING GUIDE

## THERMAL PROTECTOR

To protect from overheating, the unit has an internal thermal switch. If the internal temperature reaches a preset level, the unit will automatically shut down and the power LED will turn red. Turn the unit off for at least 40 minutes to allow the unit to cool down (pic. 32).

PROBLEM	CAUSES AND SOLUTIONS
The device does not operate.	<ul style="list-style-type: none"> <li>• Make sure that the plug is properly inserted into the device.</li> <li>• Make sure that the AC plug is properly fitted into the electrical outlet.</li> <li>• Push the ON/OFF button again.</li> </ul>

PROBLEM	CAUSES AND SOLUTIONS
The device is on but nebulizes weakly.	<ul style="list-style-type: none"> <li>• Make sure the nebulizer is seated or check for proper air tubing connection; repeat the procedure of insertion as described in the manual.</li> <li>• Make sure the nebulizer is in an upright position during use. Do not tilt beyond 30°.</li> <li>• Check whether the nebulizer is empty.</li> <li>• Clean or replace the nebulizer as described in the CLEANING AND MAINTENANCE section.</li> <li>• Check whether the air filter is dirty or obstructed. If necessary, replace it as described in the CLEANING AND MAINTENANCE section.</li> </ul>
The device is on but does not nebulize.	<ul style="list-style-type: none"> <li>• Make sure the micronization blue cone has been placed inside the nebulizer chamber.</li> </ul>
The device does not work and the light indicator is red.	<p>The thermal protector has been activated for the following reasons:</p> <ul style="list-style-type: none"> <li>• the device has been operating beyond the operating conditions indicated in this manual,</li> <li>• the device has been operating near heat sources or in hot environments.</li> </ul> <p>Let the unit cool down for at least 40 minutes, wait until the red light turns off and turn the unit on again.</p>

**Should the device not start working properly again, refer to an authorized technical service center.**

# TECHNICAL SPECIFICATIONS

<b>Model:</b>	<b>CompAir Elite</b>
Reference code:	NE-C21
Power Supply:	100-240V ~, 47-63 Hz, 12VDC, 1.25A
Max. pressure:	36 lb/sq inch
Max. air flow:	4.5 l/min
Weight:	1 lb. (with battery) - 0.5 lbs. (without battery)
Size:	(H) 119 x (L) 78 x (P) 38 mm
Noise Level:	approx. 45 dBA
Nebulizer chamber capacity:	7 ml
Class II Device (Double Safety Insulation)	
Type B Device (device with specific protection against electrical hazards)	
Device is not water proof	
Device not suitable for use in presence of anesthetic substances flammable with air, oxygen or nitrogen protoxide	
Device for intermittent use (20 min ON / 40 min OFF)	
Device not suitable for use in anesthesia or pulmonary ventila- tion systems	
Reference to Standards:	Electric Safety Standards EN 60601.1 Electromagnetic Compatibility according to EN 60601.1.2

# TECHNICAL SPECIFICATIONS

---

## AMBIENT CONDITIONS

### Storing conditions

Temperature: 50°F to 104°F (10°C to 40°C)

Humidity: 10 to 90%RH

### Operating conditions

Temperature: -13°F to 158°F (-25°C to 70°C)

Humidity: 10 to 90%RH

Altitude: da 0 a 2000m s.l.m.

## SYMBOLS

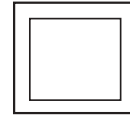
---

ON

DC

Class II

OFF

Type B

Read the  
Instructions

€ 0123

**This device fulfills the provisions of 93/42/EEC Directive.**

**Manufacturer: MEDEL S.p.A. – S. Polo di Torrile (PR) Italy**

Specifications and appearance are subject to change without notice for purposes of improvement.

# WARRANTY CERTIFICATE

---

Your Omron Nebulizer is warranted to the original retail purchaser to be free from defects in material and workmanship for a period of 5 years from the date of purchase. This warranty does not cover failures resulting from misuse, abuse, alteration of the Omron Nebulizer System, use of improper components or failure to comply with the operating instructions. This warranty does not include the nebulizer kit or accessories.

To make a warranty claim, return your Omron Nebulizer System, freight prepaid and insured (or assume the risk of loss or damage during shipment) to Omron Healthcare, Inc., 300 Lakeview Parkway, Vernon Hills, IL 60061, Attention: Service Department. All requests for warranty service should include the model number of your Omron Nebulizer System, proof of purchase (including date of purchase), your phone number and the nature of your service request. Omron will either repair or at its option replace free of charge any parts necessary to correct defects in material or workmanship covered by this limited warranty.

This limited warranty is in lieu of all other warranties, whether oral, written, express or implied. There are no warranties of merchantability or fitness for a particular purpose, except for those implied warranties arising by operation of law which are limited in duration to the 5 year term of this warranty. Omron's liability, whether based on contract, tort, warranty, strict liability, or any other theory, shall not exceed the price of the individual Omron nebulizer system which is the basis of the claim. In no event shall Omron be liable for any loss of profits, facilities, equipment or other incidental, consequential or special damages.

Some states do not allow limitations on how long an implied warranty lasts, or allow the exclusion or limitations on how long an implied warranty lasts, or allow the exclusion or limitation of incidental, consequential, or special damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights. You may also have other rights which vary from state to state.

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**Fabricante: MEDEL S.p.A. – S. Polo di Torrile (PR) Italia**