
# DEVILBISS®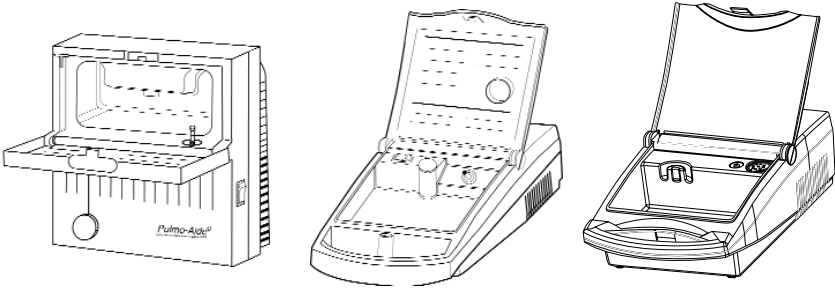

## *DeVilbiss® Compressor/Nebulizer Instruction Guide*

Pulmo-Aide® Models 5650D, 5650I, 5650H, 5650P & 5650C

PulmoMate® Models 4650D, 4650I, 4650AP, 4650L, 4650LD & 4650C

Pulmo-Aide® LT Models 3650D, 3650I, 3650P & 3650T

**CAUTION**— Federal (U.S.A.) law restricts this device to sale by, or on the order of a physician.  
Made in U.S.A.

Models 3650D, 4650D, 4650I, 5650D, 5650H, 4650AP, 4650L, 4650LD & 3650T U.L. Listed  
Models 4650C & 5650C CSA Listed

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**ENGLISH**

Important Safeguards .....	4
Model 5650H & 4650D Grounding Instructions .....	5
Introduction .....	6
Important Parts of Your DeVilbiss Compressor .....	7
Important Parts of Your Disposable Nebulizer .....	8
Important Parts of Your Reusable Nebulizer .....	8
How to Operate Your Compressor.....	8
Nebulizer Cleaning .....	10
Compressor Cleaning .....	12
Filter Change .....	12
Maintenance .....	12
Specifications .....	13
Warranty.....	14

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## IMPORTANT SAFEGUARDS

When using electrical products, especially when children are present, basic safety precautions should always be followed. Read all instructions before using. Important information is highlighted by these terms:

**DANGER**– Urgent safety information for hazards that will cause serious injury or death.

**WARNING**–Important safety information for hazards that might cause serious injury.

**CAUTION**– Information for preventing damage to the product.

**NOTE**– Information to which you should pay special attention.

### READ ALL INSTRUCTIONS BEFORE USING.

#### DANGER

##### To reduce the risk of electrocution:

1. Always unplug this product immediately after using.
2. Do not use while bathing.
3. Do not place or store product where it can fall or be pulled into a tub or sink.
4. Do not place in or drop into water or other liquid.
5. Do not reach for a product that has fallen into water. Unplug immediately.

#### WARNING

##### To reduce the risk of burns, electrocution, fire or injury to persons:

1. A product should never be left unattended when plugged in.
2. Close supervision is necessary when this product is used by, on, or near children or invalids.
3. Use this product only for its intended use as described in this guide. Do not use attachments not recommended by the manufacturer.
4. Never operate this product if:
  - a. It has a damaged power cord or plug.
  - b. It is not working properly.
  - c. It has been dropped or damaged.
  - d. It has been dropped into water.
 Return the product to an authorized Sunrise service center for examination and repair.
5. Keep the power cord away from heated surfaces.
6. Never block the air openings of the product or place it on a soft surface, such as a bed or couch, where the air openings may be blocked. Keep the air openings free of lint, hair, and the like.
7. Never use while drowsy or asleep.
8. Never drop or insert any object into any opening or hose.
9. Do not use outdoors. This product is for household use only.

10. Do not use in an oxygen enriched environment.

11. Connect this product (for grounded models) to a properly-grounded outlet only. See *Grounding Instructions*.

**NOTE**– This unit employs a thermal limiter that will shut off the entire product should the motor overheat.

5650–The unit will restart when the motor cools to an acceptable level.

4650/3650–Contact your qualified Sunrise provider for a replacement.

If repair or replacement of the cord or plug is necessary, contact your qualified Sunrise provider.

This appliance has a polarized plug (one blade is wider than the other). As a safety feature, this plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug. If it still does not fit, contact a qualified electrician. Do not attempt to defeat this safety feature.

### MODEL 5650H & 4650D GROUNDING INSTRUCTIONS

This product should be grounded. In the event of an electrical short circuit, grounding reduces the risk of electric shock by providing an escape wire for the electric current. This product is equipped with a cord having a grounding wire with a grounding plug. The plug must be plugged into an outlet that is properly installed and grounded.

#### DANGER


##### Improper use of the grounding plug can result in the risk of electric shock.

If repair or replacement of the cord or plug is necessary, contact your qualified Sunrise provider.

1. This unit is for use on a nominal 115V circuit and has a grounding plug. A temporary adapter, which looks like the adapter illustrated in Figures B and C, may be used to connect this plug to a 2-pole receptacle (as shown in Figure B) if a properly grounded outlet is not available. The temporary adapter should be used only until a qualified electrician can install a properly grounded outlet (Figure A). The green colored rigid ear, lug, and the like extending from the adapter must be connected to a permanent ground such as a properly grounded outlet box cover. Whenever the adapter is used, it must be held in place by the metal screw.

- If it is necessary to use an extension cord, use only a 3-wire extension cord that has a three-blade grounding plug, and a 3-slot receptacle that will accept the plug on the unit. If power cord is damaged do not use unit, and contact a qualified Sunrise Medical provider.

## SAVE THESE INSTRUCTIONS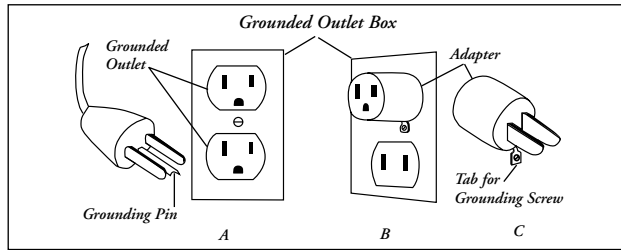

## INTRODUCTION

Your doctor has prescribed a liquid medication to treat your respiratory condition. To best utilize this liquid medication, he has prescribed a DeVilbiss brand compressor/nebulizer. Your DeVilbiss compressor/nebulizer works to convert the medication to a high-quality mist of fine particles that penetrates deep into the lungs.

Be sure that you read and understand the information in this instruction guide. By following these simple instructions and the advice of your physician, your compressor will become an effective addition to your therapeutic routine.

### Statement of Intended Use

The DeVilbiss compressor/nebulizer is an AC powered air compressor that provides a source of compressed air for home health care use. The product is used in conjunction with a jet (pneumatic) nebulizer to convert liquid medication into an aerosol form with particles smaller than 5 microns diameter for inhalation into the patient's respiratory tract for the treatment of respiratory disorders.

The target population for this device consists of both adult and pediatric patients suffering from, but not limited to asthma, cystic fibrosis, and chronic obstructive pulmonary disease. Further, additional applications for aerosolized medication are constantly under investigation and this device may be deemed suitable for such applications as prescribed.

The intended environment for use of the product is in the patient's home on the order of a physician.

## IMPORTANT PARTS OF YOUR DEVILBISS COMPRESSOR (FIGURES A, B & C)

- Storage compartment
- Storage compartment door with operating instructions
- Power switch
- Filter cap
- Air-outlet connector
- Carrying handle
- Power cord storage
- Nebulizer holder

**NOTE**— Models 4650D & 5650H are equipped with a three-prong plug and must be grounded. All other models are equipped with a two-prong plug.

### Compressor Figures

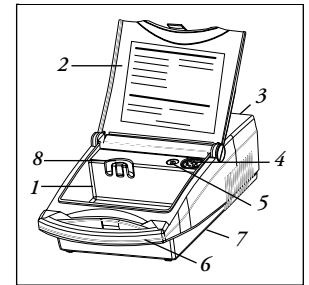
Figure A— 3650

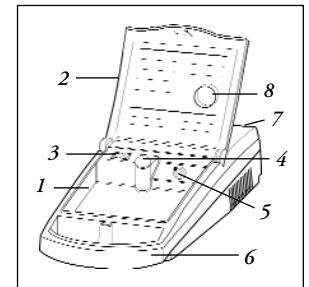
Figure B— 4650


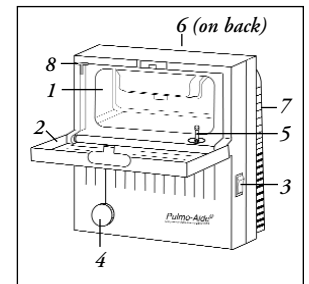
Figure C— 5650

### IMPORTANT PARTS OF YOUR DISPOSABLE NEBULIZER

1. Mouthpiece
2. T-piece
3. Cap
4. Medication Cup
5. Nebulizer Air-Inlet Connector
6. Baffle
7. Tubing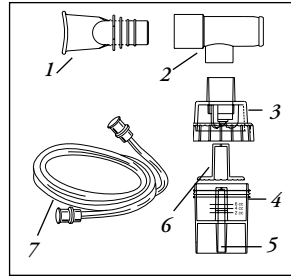


Disposable Nebulizer

Nebulizer parts are not sold separately. Additional nebulizers (part #4650D-620) are available through your Sunrise provider.

**NOTE**– Nebulizer received may not be the one shown.

### IMPORTANT PARTS OF YOUR REUSABLE NEBULIZER

1. Mouthpiece
2. Cap
3. Baffle
4. Medication Cup
5. Nebulizer Air-Inlet Connector
6. Interrupter Adapter (if applicable)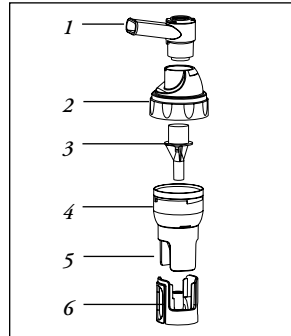

Reusable Nebulizer

Nebulizer parts are not sold separately. Additional nebulizers (Part #s 800D/800C/800G) are available through your Sunrise provider.

**NOTE**– Nebulizer received may not be the one shown.

### HOW TO OPERATE YOUR COMPRESSOR

**NOTE**– Before initial operation, your nebulizer should be cleaned following cleaning instructions, or as recommended by your physician or Sunrise provider.

1. Place compressor on a level, sturdy surface so that the controls can be easily reached when seated.
2. Open door to storage compartment (**Fig. 1**).
3. Make sure the power switch is in the "Off" position (**Fig. 2**). Unwrap power cord and plug power cord into an appropriate wall outlet (refer to *Specifications*). Plug models 5650H & 4650D into a grounded wall outlet (refer to *Grounding Instructions*).

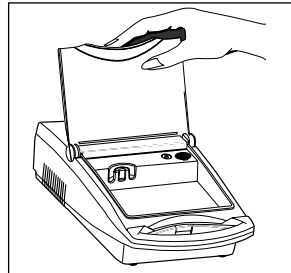
Figure 1

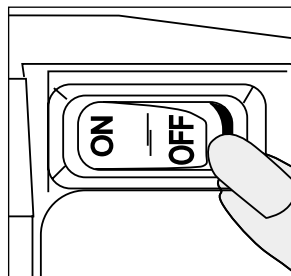
Figure 2

### ⚠ DANGER

DeVilbiss compressor/nebulizer must be operated on the specified power source to avoid risk of electric shock and damage to the compressor.

4. Wash hands.
5. Connect one end of the nebulizer tubing to the compressor air-outlet connector (**Fig. 3**).

**NOTE**– During high-humidity weather conditions, condensation (water build up) can occur in the internal hose of the compressor. It is recommended that the compressor be turned on and allowed to run for two (2) minutes before the tubing is attached to the air-outlet connector.

6. Assemble clean nebulizer parts by placing baffle down in medication cup. Holding cup stationary, screw on nebulizer cap. Add prescribed medication through the opening on cap using a medicine dropper or pre-measured dose container (**Fig. 4**).
7. Assemble mouthpiece and t-piece (if applicable) and insert into the top of the nebulizer cap (**Fig. 5**). If using an aerosol mask, insert the bottom part of the mask directly into the top of the nebulizer cap.
8. Attach tubing to nebulizer air-inlet connector (**Fig. 6**).

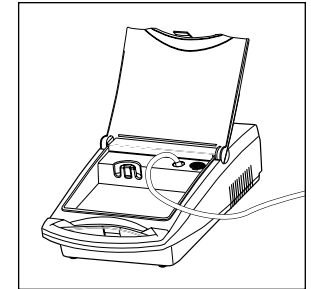
Figure 3

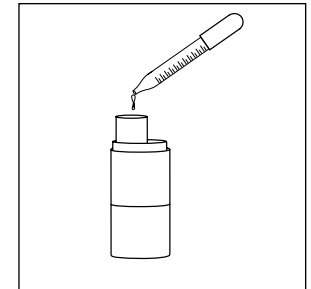
Figure 4

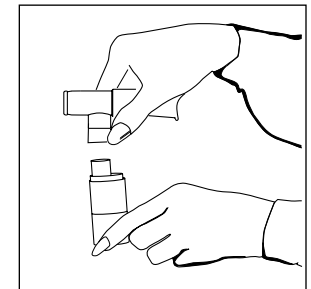
Figure 5

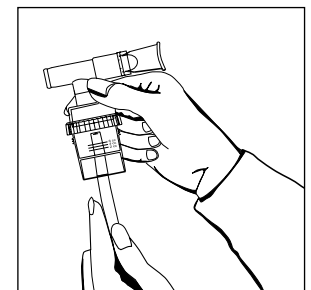
Figure 6

9. Press the power switch "On" to start the compressor.
10. Begin treatment by placing the mouthpiece between teeth. With mouth closed, inhale deeply and slowly through mouth as aerosol begins to flow, then exhale slowly through the mouthpiece (Fig. 7). If treatment needs to be interrupted, simply press power switch "Off".

**NOTE**– Some physicians recommend a "clearing breath" after every five to seven treatment breaths. Remove mouthpiece from mouth and hold breath for at least five seconds (ten is better), then exhale slowly.

11. If an aerosol mask is used, place the mask over mouth and nose (Fig. 8). As aerosol begins to flow, inhale deeply and slowly through mouth, then exhale slowly.

## NEBULIZER CLEANING

All parts of the nebulizer, except tubing, should be cleaned according to the following instructions. Your physician and/or Sunrise provider may specify a certain cleaning procedure. If so, follow their recommendations.

### **⚠WARNING**

**To prevent possible risk of infection from contaminated medication, cleaning of the nebulizer is recommended after each aerosol treatment. Disinfecting is recommended once a day.**

#### **Clean After Every Use:**

1. With power switch in the "Off" position, unplug power cord from wall outlet.
2. Disconnect tubing from the air-inlet connector and set aside.
3. Disassemble mouthpiece or mask from cap. Open nebulizer by turning cap counterclockwise and removing baffle.
4. Wash all items, except tubing, in a hot water/dishwashing detergent solution. Rinse under hot tap water for 30 seconds to remove detergent residue. Allow to air dry.

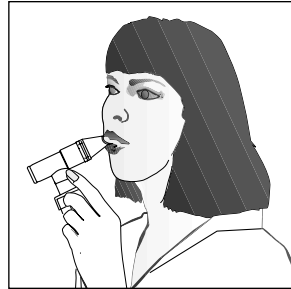
Figure 7

Figure 8

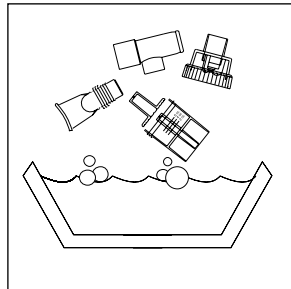
Figure 9

#### **Disinfect Daily:**

1. Using a clean container or bowl, soak items in three parts hot water to one part white vinegar for 30 minutes (Fig. 9) OR use a medical bacterial-germicidal disinfectant available through your provider. For reusable nebulizer only, clean daily in dishwasher using top shelf. Be sure to follow manufacturer's instructions carefully.
2. With clean hands, remove items from disinfectant solution, rinse under hot tap water, and air dry on a clean paper towel. Store in a zip-lock bag.

**NOTE**– Do not towel dry nebulizer parts; this could cause contamination.

**CAUTION**– The DeVilbiss Reusable Nebulizer is dishwasher safe, but do not place any disposable nebulizer parts in an automatic dishwasher; doing so could cause damage.

### **⚠WARNING**

**To prevent possible risk of infection from contaminated cleaning solutions, always prepare fresh solution for each cleaning cycle and discard solution after each use.**

3. Keep the outer surface of the tubing dust-free by wiping regularly. Nebulizer tubing does not have to be washed because only filtered air passes through it.
- NOTE**– The DeVilbiss Disposable Nebulizer will last at least 15 days and possibly longer, depending on usage. Proper cleaning will also help extend the life of the nebulizer. Because it is disposable, we recommend that an extra nebulizer be kept on hand at all times. Sunrise also manufactures the DeVilbiss Reusable Nebulizer that is dishwasher safe and may be cleaned and reused for up to one year.

## COMPRESSOR CLEANING

1. With power switch in the "Off" position, unplug power cord from wall outlet.
2. Wipe outside of the compressor cabinet with a clean damp cloth every few days to keep dust-free.

**CAUTION**– Do not submerge in water; doing so will result in compressor damage.

## FILTER CHANGE

1. Filter should be changed every 6 months or sooner if filter turns completely gray in color.
2. Remove filter cap by grasping it firmly and pulling out of the unit (**Figs. A, B or C**).
3. Remove dirty filter with fingers and discard.
4. Replace with a new DeVilbiss filter (part #5650D-602). Additional filters should be purchased from your Sunrise provider.
5. Push filter cap with new filter into position.

**CAUTION**– Reusing filter or substituting any other material such as cotton for a DeVilbiss air-inlet filter will result in compressor damage.

**NOTE**– An extra filter is packaged with the 4650 & 5650 series units.

## MAINTENANCE

All maintenance must be performed by a qualified Sunrise provider or authorized service center.

### DANGER

**Electric shock hazard. Do not remove compressor cabinet. All disassembly and maintenance must be done by a qualified Sunrise provider.**

## SPECIFICATIONS

<b>Size</b> 3650 Series .....	7.7" W x 4.1" H x 13" D (19.5 cm x 10 cm x 33 cm)
4650 Series .....	7" W x 3.8" H x 13" D (18 cm x 10 cm x 33 cm)
5650 Series .....	10.1" W x 10.5" H x 6.5" D (26 cm x 27 cm x 17 cm)

<b>Weight</b> 3650 Series .....	5.4 lbs. (2.45 kg)
4650 Series .....	5.3 lbs. (2.40 kg)
5650 Series .....	7.1 lbs. (3.2 kg)

### Electrical

<b>Requirements</b> 3650D & 3650T .....	2.3 amps, 115 VAC 60 Hz
3650I .....	1.4 amps, 230 VAC 50 Hz
3650P .....	1.4 amps, 230 VAC 60 Hz
4650D, 4650I, 4650AP, 4650L, 4650LD & 4650C .....	2.5 amps, 115 VAC 60Hz
4650I .....	1.4 amps, 230 VAC 50 Hz
5650D, 5650H & 5650C .....	1.3 amps, 115VAC 60Hz
5650I .....	0.8 amps, 230VAC 50Hz
5650P .....	0.8 amps, 230VAC 50/60Hz

<b>Sound Level</b> 3650 Series .....	58 dBA
4650 Series .....	53 dBA
5650 Series .....	51 dBA

<b>Power Consumption</b> 3650D & 3650T .....	140 watts maximum
3650I & 3650P .....	160 watts maximum
4650D, 4650I, 4650AP, 4650L, 4650LD & 4650C .....	140 watts max.
4650I .....	160 watts maximum
5650D, 5650H & 5650C .....	90 watts maximum
5650I & 5650P .....	100 watts maximum

### Max. Compressor

<b>Pressure</b> 3650 & 4650 series .....	35 psig or greater*
5650 series .....	30 psig or greater*

<b>Compressor Free Air Flow</b> (3650 & 4650) .....	8 lpm or greater*
(5650) .....	9 lpm or greater*

**Nebulizer Operating Pressure** all models .....

**Nebulizer Liter Flow** all models .....

**Nebulizer Capacity** all models .....

**Nebulizer Nebulization Rate** all models .....

**MMAD** all models .....

\*Conditions may vary based on altitude above sea level, barometric pressure changes.

**FIVE-YEAR LIMITED WARRANTY ON COMPRESSOR PARTS**

All DeVilbiss Compressor models are warranted to be free from defective workmanship and material for a period of five years from the date of purchase. Any defective part(s) will be repaired or replaced at Sunrise’s option if the unit has not been tampered with or used improperly during that period. Make certain that any malfunction is not due to inadequate cleaning or failure to follow the instructions. If repair is necessary, contact your authorized Sunrise provider or Sunrise Service Department at 800-333-4000 (814-443-4881) (Canada 800-263-3390 or 905-660-2459) for instructions. NOTE— This warranty does not cover providing a loaner compressor, compensating for costs incurred in compressor rental while said unit is under repair, or costs for labor incurred in repairing or replacing defective part(s).

THERE IS NO OTHER EXPRESS WARRANTY. IMPLIED WARRANTIES, INCLUDING THOSE OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO THE DURATION OF THE EXPRESS LIMITED WARRANTY AND TO THE EXTENT PERMITTED BY LAW ANY AND ALL IMPLIED WARRANTIES ARE EXCLUDED. THIS IS THE EXCLUSIVE REMEDY AND LIABILITY FOR CONSEQUENTIAL AND INCIDENTAL DAMAGES UNDER ANY AND ALL WARRANTIES ARE EXCLUDED TO THE EXTENT EXCLUSION IS PERMITTED BY LAW. SOME STATES DO NOT ALLOW LIMITATIONS ON HOW LONG AN IMPLIED WARRANTY LASTS, OR THE LIMITATION OR EXCLUSION OF CONSEQUENTIAL OR INCIDENTAL DAMAGES, SO THE ABOVE LIMITATION OR EXCLUSION MAY NOT APPLY TO YOU.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

**MANUFACTURER'S NOTE**

Thank you for choosing a DeVilbiss compressor/nebulizer. We want you to be a satisfied customer. If you have any questions or comments, please send them to our address on the back of this manual.

<p>FOR SERVICE CALL YOUR AUTHORIZED SUNRISE PROVIDER:</p> <p>_____</p> <p>_____</p> <p>_____</p> <p>Phone _____</p> <p>Purchase Date _____</p> <p>Serial # _____</p>
--

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©2001, Sunrise Medical 5.01
**Sunrise Medical Ltd.**
**Sunrise Medical Pty. Limited**