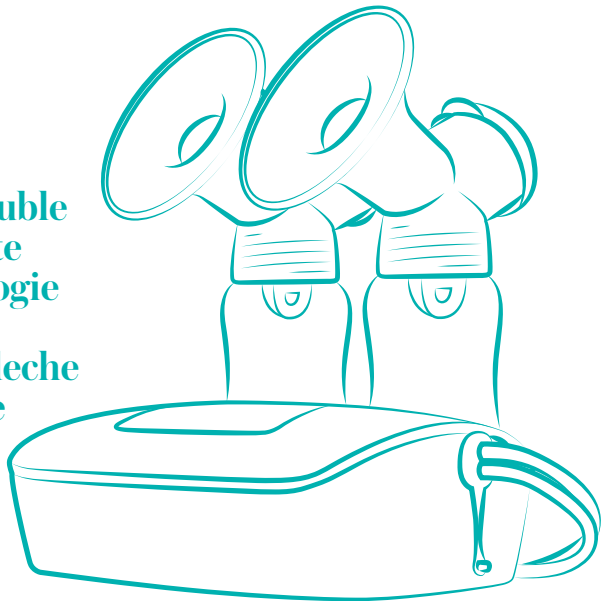
Advanced Double Electric Breast Pump

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For video instructions about how to use and care for your pump,
visit: www.evenflofeeding.com

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Thank you for choosing Evenflo Feeding to support you on your breastfeeding journey.

If you have questions about the performance or operation of this pump, please contact our Parent’s Ally Resource Center.

For more information about breastfeeding or pumping, contact a lactation expert, your physician, or visit one of the websites listed below.

USA/Canada: 1-855-EFI-BABY (334-2229) or visit us online at www.evenflofeeding.com.

International Lactation Consultant Association

Lactation consultants with the IBCLC credential specialize in the clinical management of breastfeeding. There may be a fee to use lactation consultant services. To locate one near you, call: (919) 861-5577. www.ilca.org

La Leche League

Provides free mother-to-mother breastfeeding support.
Phone: 1-800-LALECHE (US), or (847) 519-7730. www.llli.org

Office on Women’s Health

Provides trained breastfeeding peer counselors for free telephone help with common issues. Phone: 1-800-994-9662. www.womenshealth.gov

WIC (Women, Infant and Children Program)

Provides breastfeeding help to mothers and children enrolled in the WIC program. www.fns.usda.gov

Babies in the Workplace

www.babiesatwork.org

Low Milk Supply Information and Support

For mothers experiencing low milk supply.
www.lowmilksupply.org

Breastfeeding USA

Mother to mother breastfeeding support.
www.breastfeedingusa.org

Breast and Bottle Feeding

Support for all parents in their infant feeding journey—be it exclusively breast, exclusively bottle, with breast milk, formula, or any combination.
www.breastandbottlefeeding.com

Please Note: This pump is a personal care item and should never be shared between mothers, resold, or given from one mother to another. Once the package has been opened, this product must not be returned to the store.

Indications for Use: The Evenflo Advanced Double Electric Breast Pump is an electrically powered suction device intended to express and collect milk from a lactating woman’s breasts. This pump is intended for a single user.

Please note that this pump should not be serviced or disassembled by the consumer. If you have any issues or concerns with the product, please contact our Parent’s Ally Resource Center.

Evenflo Feeding has established a robust support program for this pump, including 1:1 pumping support by phone, with the goal of making your pumping success our top priority.

After working with our Parent’s Ally support team, our experts will determine what solutions we can offer, in accordance with our policies. For full warranty replacement options to be available, you must have proof of at least one of the following: (1) a purchase receipt, (2) proof of delivery through insurance, or (3) prior registration of your pump at www.registermypump.com. Refunds will only be available with a purchase receipt dated in the last 6 months.

Table of Contents

Important Safeguards and Warnings	2
Identification of Parts	4
Before First Use	5
Assembling the Flange	7
Single Versus Double Pumping	8
Attaching Flange Assembly to Pump Unit	8
Power Operations	8
Using Batteries	9
Using the AC Adapter	10
Check the Fit of the Flange	10
Using Your Breast Pump	11
After Every Use	14
Storing Your Breast Milk.....	14
Using Your Breast Milk	15
Troubleshooting	16
Limited Warranty	17

⚠️ IMPORTANT SAFEGUARDS AND WARNINGS

Read all instructions before using and save for future use.

When using this pump and other electrical products, basic safety precautions should always be maintained, including the following:

DANGER: To reduce the risk of burns, electrocution, fire or injury to persons:

General Usage Safeguards & Warnings

- **ALWAYS** unplug this product immediately after using.
- **DO NOT** use while bathing.
- **DO NOT** place or store this product where it can fall or be pulled into a tub or sink.
- **DO NOT** place in or drop into water or other liquid.
- **DO NOT** reach for an electrical product that has fallen into water. Unplug the product immediately.
- **NEVER** immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven.
- Use this product only for its intended use as described in these instructions.
- Breast pumps are personal care items, for one user only.
- This product should never be left unattended when plugged in.
- **NEVER** use while sleepy or drowsy.
- Close supervision is necessary when this product is used near children.
- **DO NOT** allow small children or pets to play with the pump unit, flange, AC adapter, etc. These parts may pose a choking or strangulation hazard.
- Pump contains small parts which could cause a child to choke. Keep pump parts and accessories out of children's reach.
- **DO NOT** use attachments not recommended by the manufacturer.
- **DO NOT** use this product if it is broken or any parts are missing.
- **NEVER** drop or insert any object into any opening of the pump unit.

Flange Usage Safeguards & Warnings

- **DO NOT** use flange insert if your nipple is larger than the flange insert's center opening. See page 11 for proper flange fit.
- If you are experiencing engorgement, prevent possible injury by discontinuing use of this pump and consulting a medical provider.

⚠️ IMPORTANT SAFEGUARDS AND WARNINGS

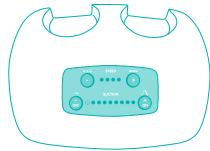
Pump Power Source Safeguards & Warnings

- **NEVER** operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, dropped into water, or if any parts are broken or missing.
- Keep the cord away from heated surfaces.
- **DO NOT** use outdoors with the AC adapter or operate where aerosol spray products are being used, or oxygen is being administered.
- Operate pump using **ONLY** 6 "AA" name-brand batteries (not included) or with the Evenflo AC adapter provided, and plug the adapter into a standard wall outlet.
- Operate the AC adapter on 100 – 240 volts 50/60 Hz AC only. Connect only to household outlet power supplies of this description. For use in countries outside North America, a plug converter (not included) may be required.
- Protect power cord and AC adapter cord. Route cords so that they are not walked on or pinched by items placed upon them.
- **DO NOT**, under any circumstances, use this AC adapter with any other equipment. Likewise, do not use the adapter from any other product on this breast pump.
- **DO NOT** mix old and new batteries. **DO NOT** mix alkaline, standard (carbon-zinc), or rechargeable batteries; leakage, explosion, and/or fire could occur.
- Remove batteries before storing unit for a prolonged period of time.


Breast Milk Safeguards & Warnings

- **DO NOT** heat breast milk in the microwave. Microwaving can destroy nutrients in breast milk and can heat liquids unevenly resulting in burns or scalds. See page 15 for proper use of breast milk.
- **DO NOT** re-freeze breast milk. See page 14 for proper milk storage.
- **ALWAYS** test temperature of liquid before feeding.
- If you are a mother who is infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), pumping breast milk will not reduce or remove the risk of transmitting the virus to your baby through your breast milk.

Save These Instructions for Future Use.

Pump Unit

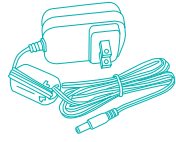
AC Adapter

Tubing Adapter

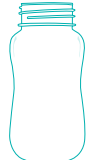Tubing

Bottle

Bottle Cap Ring

Sealing Disc

Flange
(30.5 mm)

Valve

Membrane

Diaphragm
Cap

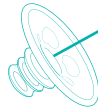
Diaphragm

24.5-mm Flange Insert

28.0-mm Flange Insert

Included flange inserts vary by kit.
Additional sizes available separately, see pages 10-11.

Universal Shoulder Bag
(Some Models Only)

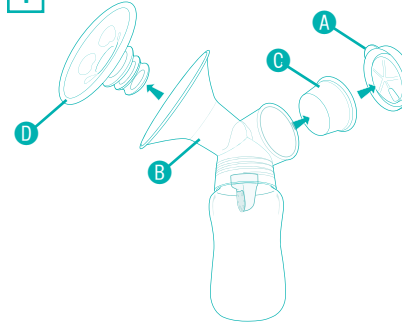
Ice Pack
(Some Models Only)

Cooler Bag
(Some Models Only)


IMPORTANT: Before using your breast pump, disassemble and sanitize thoroughly. Always wash hands well with soap and water before handling your pump.

1


Remove diaphragm cap **A** from flange assembly **B** by pulling up on cap tab. Remove diaphragm **C** and flange insert **D** from flange assembly.

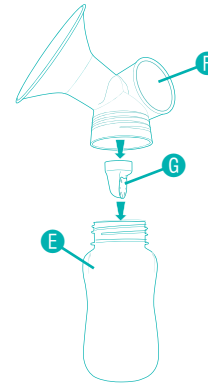2

Unscrew bottle **E** from bottom of flange **F**. Remove valve assembly **G** from the bottom of the flange **F**.

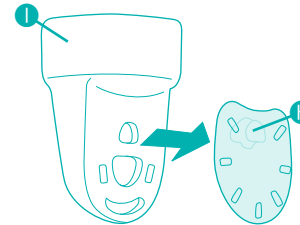3


Remove membrane **H** from valve **I**.

Repeat steps 1 through 3 for second flange disassembly.

Clean Your Pump Parts

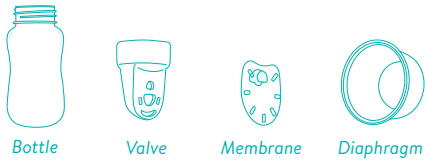PARTS THAT MAY BE SANITIZED:

To Sanitize Parts on the Stove:

Place disassembled pump parts in a pan of boiling water for 5 minutes. Stir frequently. DO NOT allow parts to rest on bottom of pan as they may melt or become damaged.

To Sanitize Parts in the Microwave:

Please follow sanitation device instructions.

PARTS THAT ARE HAND WASH ONLY:

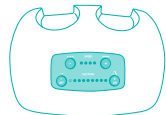
Diaphragm Cap

IMPORTANT

DO NOT BOIL OR SANITIZE IN MICROWAVE. Boiling or sanitizing in the microwave will damage these parts and the PUMP WILL NO LONGER WORK.

DO NOT WASH:

Pump Unit

AC Adapter

Tubing Adapter

Tubing

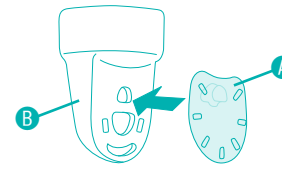
WARNING

NEVER immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven. Damage to parts and oven may occur. Do not boil or wash tubing or tubing adapter. Wipe the pump unit with a clean, damp cloth as needed.

Before Every Use

- Wash your hands with soap and water before handling your pump.
- Ensure parts are clean and dry. Refer to cleaning instructions on page 14.
- Use care when handling the valve/membrane assembly to avoid damage.
- Visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the damaged parts have been replaced.


1

Attach the membrane **A** to the valve **B**. Ensure membrane can move by gently lifting the membrane away from the valve. The membrane should return to lie flat against the valve.


Note: If the membrane is not lying flat against the valve, the pump will lose suction and will not work properly. Also, make sure the membrane is not torn, as this too will cause decreased suction and the pump not to work properly.

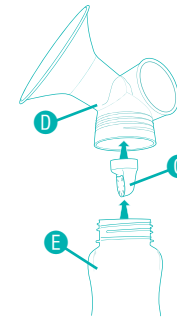2

Push valve assembly **C** onto the bottom of the flange **D**. Screw bottle **E** into bottom of flange.

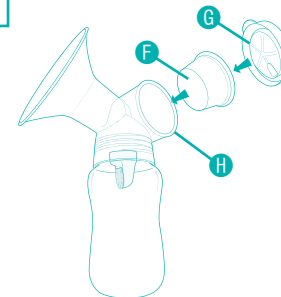
3

Install diaphragm **F** and diaphragm cap **G** on to flange assembly **H**.

Note: Diaphragm cap can be turned to orient tubing as desired.

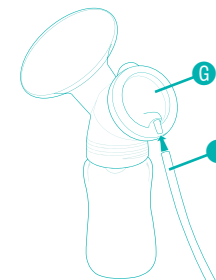4


Attach tubing **I** to diaphragm cap **G**.

Note: If you intend to pump from both breasts at the same time, repeat steps 1 through 4 for the second flange.

Single Versus Double Pumping

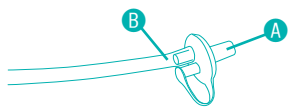
The tubing adapter can be used to pump with one or both flanges.

To Pump One Breast at a Time:

Cap one side of the tubing adapter with the attached cap **A**. Connect one tube **B** to the other side.

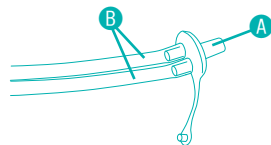
Note: The open side must be completely capped to ensure proper suction.

To Pump Both Breasts at the Same Time:

Ensure that both tubes **B** are properly connected to the tubing adapter **A**.

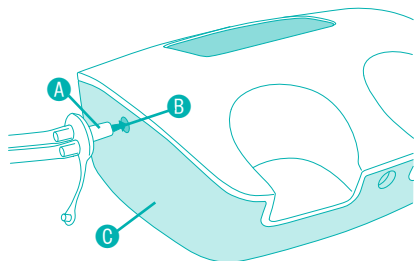Attaching Flange Assembly to Pump Unit

Insert tubing adapter **A** into the port **B** on the side of the pump unit **C**.

Note: To minimize damage to the pump unit, always operate with a flange assembly attached.

Power Options

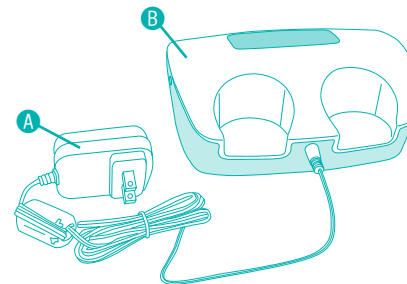
This pump can be powered by: the supplied Evenflo AC adapter, six “AA” brand name batteries, or the Evenflo Advanced Car Adapter (sold separately).

Note: When batteries are first installed or when adapter is first plugged in, the pump unit will briefly turn on then off; this is normal.

Using the AC Adapter


⚠ WARNING

- **DO NOT** under any circumstances, use this AC adapter with any other equipment. Likewise, **DO NOT** use the adapter from any other product on this breast pump. If you lose the AC adapter, contact Evenflo for a replacement.
- **ALWAYS** remove plug from product and wall by pulling on the plug/housing. **DO NOT** pull on the cord as this could damage the cord.
- **NEVER** operate this product if it has a damaged plug or cord.

Before using the AC adapter **A**, remove batteries from pump unit. Plug AC adapter into the pump unit **B** then plug the adapter into 100 – 240V AC power only (standard household outlet). For use in countries outside of North America, a plug converter (not included) may be required. Unplug the adapter from the outlet after each use.

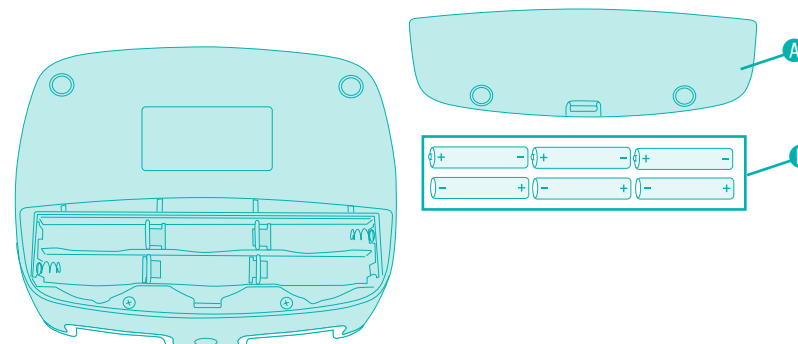Using Batteries


⚠ WARNING

- **DO NOT** mix old and new batteries.
- **DO NOT** mix battery types; leakage, explosion and/or fire could occur.
- Remove batteries before putting product into storage.

If you plan to use batteries, lift tab on battery door **A** and insert six alkaline “AA” batteries **B** (not included). For best performance, brand name non-rechargeable batteries are recommended. Generic batteries may impact pump performance. Replace battery door.

Low Battery Indication

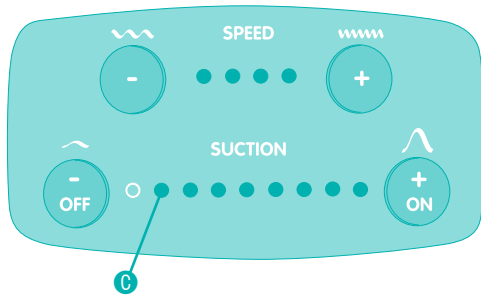When batteries are low, the first suction level indicator **C** will blink, but the pump will still be usable.

Once battery level is too low to pump, the pump will stop and not turn on until the batteries are replaced or the AC adapter is used. During this time the first suction level indicator **C** will blink, but the pump will not turn on.

Check the Fit of the Flange

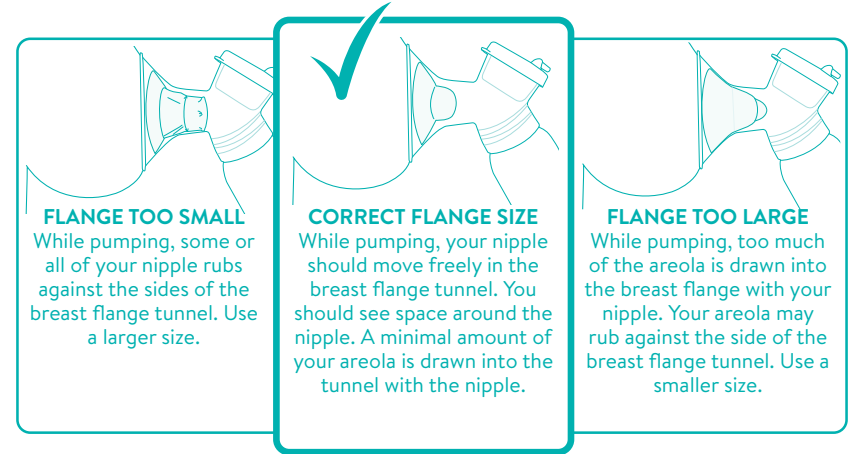
The Advanced Double Electric Pump works with the Evenflo AdvancedFit™ Flange System. Additional sizes are available separately for purchase, allowing you to find your best fit.

The AdvancedFit™ Flange System available sizes include:

- X-Small = 21.0mm (insert)
- Small = 24.5mm (insert)
- Medium = 28.0mm (insert)
- Large = 30.5mm (no insert)
- X-Large = 33.0mm (insert)
- XX-Large = 36.0mm (no insert)

Correct Flange Size vs. Too Small or Too Large


A proper flange size is needed for comfort and to maintain your milk supply. Your flange size can change several times throughout your journey. Recheck your flange fit frequently to ensure you are using the proper flange size.

IMPORTANT:

- **DO NOT** use the flange insert if your nipple is larger than the center opening.
- Your breast may change over time. Should this occur, choose a different size for maximum comfort.
- **Included flange sizes may not fit all breast and nipple sizes, but additional sizes can be purchased separately.** If you are having issues after trying all included sizes, please contact our Parent’s Ally Resource Center for assistance.

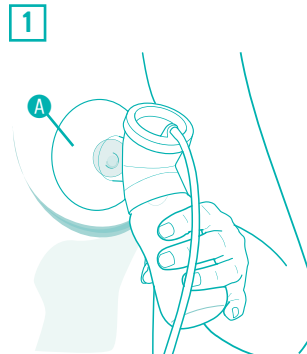
Using Your Breast Pump

To ensure a good seal, place the flange **A** gently, but firmly, against your breast, with your nipple directly in the center, as shown. Hold the bottle upright to ensure that milk flows into the bottle. Use the flange or appropriate flange insert that is the most comfortable and seals against your breast the best.

Note: Use only one soft flange insert per flange. Pressing the flange too hard against your breast can restrict milk flow.

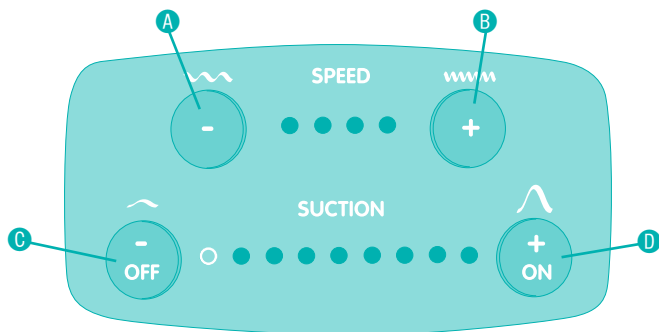2

Using The Controls

Turn on the pump by pressing the “ON” button **D**.

The pump will turn on at the **highest speed** and **lowest suction** setting to help stimulate your first letdown (milk release).

When milk starts to flow, **decrease speed A** and **increase suction D** to the highest comfortable setting. If uncomfortable, turn suction down slightly.

When milk flow slows, **increase speed B** and **maintain suction** setting. When milk begins to flow again, **decrease speed A** halfway. If desired, repeat this step for additional letdowns.

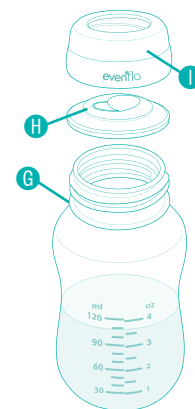
Turn off the pump. After breast milk has been expressed, decrease suction **C** to the lowest setting. Release button, then press and hold the “OFF” button **C** until pump turns off (approximately 1 to 2 seconds). If necessary, slide a finger between the flange and your breast to break any remaining suction, then remove the flange.

TIP: Some users find that using the highest speed with the highest comfortable suction throughout the pumping session yields the most amount of milk in the shortest amount of time.

IMPORTANT: If you feel pain in your breast or nipple, turn the pump off (see above) or slide a finger between the flange and your breast to immediately break the suction. If you feel discomfort at every suction level, turn the pump off and contact your healthcare provider for assistance.


NOTE: Pump will automatically shut off after 40 minutes. Do not overfill the 5-ounce collection bottle; when freezing your milk for storage, space is needed in the bottle for the milk to expand.

3

Carefully unscrew bottle **G** from flange. You may now transfer milk into your preferred milk storage container, such as breast milk storage bags, or you can store your milk directly in the milk collection bottle. Place the sealing disc **H** into the cap ring **I** and screw onto the bottle securely. (see page 14 for information on storing breast milk)

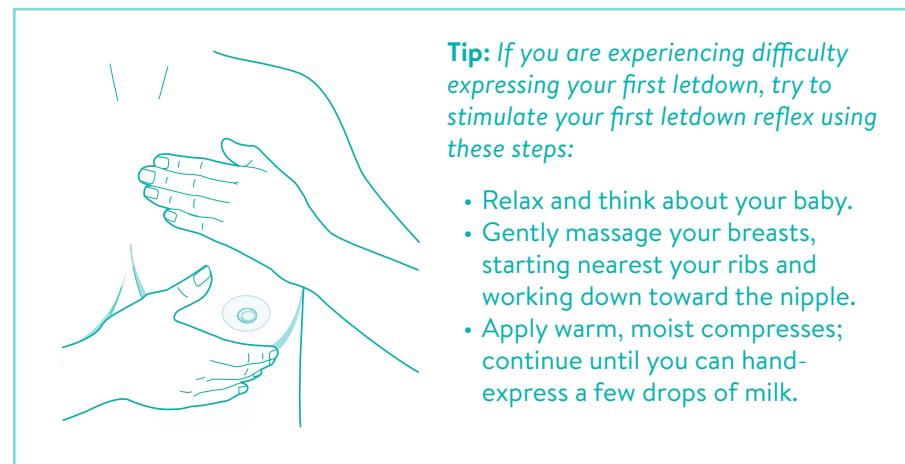Note: When using as a single pump on one breast at a time, repeat steps 1 through 3 on your other breast.

Tip: If you are experiencing difficulty expressing your first letdown, try to stimulate your first letdown reflex using these steps:

- Relax and think about your baby.
- Gently massage your breasts, starting nearest your ribs and working down toward the nipple.
- Apply warm, moist compresses; continue until you can hand-express a few drops of milk.

For optimal performance, follow these steps after every use:

- **Completely disassemble the flange assembly, see page 5.**
 - Rinse bottles, diaphragms, valves, membranes, flanges, and flange inserts in cool water to remove milk residue.
 - Hand wash with warm, soapy water.
 - Rinse thoroughly and air dry before assembly. You may also wash your flange assembly (after rinsing) on the top rack of your dishwasher.
 - Use care when handling the valve/membrane assembly during cleaning to avoid damage. Before each use, visually inspect the individual components for cracks, chips, tears, discoloration or deterioration. In the event that damage to the device is observed, please discontinue use until the parts have been replaced.
- Wipe the pump unit with a clean, damp cloth as needed.
- Inspect the tubing for condensation or milk after each use. If condensation is observed in the tubing, contact Evenflo for replacement parts.

WARNING

NEVER immerse pump unit or AC adapter in water or attempt to sanitize in a microwave oven. Damage to parts and oven may occur. **DO NOT** boil or wash tubing or tubing adapter.

Storing Your Breast Milk

1. After sealing your bottle tightly with the sealing disc and cap ring, label the bottle with the date and time the milk was expressed.
2. Use the chart below as a guide for storing your breast milk for future use.
Note: In hot weather, chill milk immediately to prevent spoilage.

Breast Milk Storage Guidelines*			
	Room Temperature	Refrigerator	Freezer
Freshly Expressed Breast Milk	Up to 4 hours	Up to 4 days	Within 6 months is best Up to 12 months is acceptable
Thawed Breast Milk (previously frozen)	1 - 2 hours	24 hours	NEVER re-freeze milk after it has thawed

* CDC. Proper Storage and Preparation of Breast Milk, Revised 2020. https://www.cdc.gov/breastfeeding/recommendations/handling_breastmilk.htm

* ABM Clinical Protocol #8: Human Milk Storage Information for Home Use for Full-Term Infants, Revised 2017. <https://abm.memberclicks.net/assets/DOCUMENTS/PROTOCOLS/8-human-milk-storage-protocol-english.pdf>

Note: Breast milk should be stored in the refrigerator or freezer compartment away from the door to avoid changes in temperature.

For models that include a cooler bag:

1. Place frozen ice packs in the included cooler bag.
2. Place the sealed bottle(s) inside and zip the bag closed.
3. When frozen ice packs are used and the cooler bag is kept at room temperature of 68 – 77° F (20 – 25° C), breast milk can be safely stored for up to 10 hours.

Note: Avoid storing the cooler bag in a hot closet, cabinet, or car.

Using Your Breast Milk

1. Thaw frozen milk slowly in the refrigerator. Avoid letting milk sit out at room temperature to thaw. (For quicker thawing, hold container under running water – start with cool water and gradually increase the temperature.)
2. Just before feeding, warm refrigerated milk:
 - Use a bottle warmer, or
 - Hold the sealed bottle under warm, running water, or
 - Heat water in a cup or other small container, then place sealed bottle in the water to warm.
 - DO NOT heat breast milk for more than 15 minutes.
3. The milk fat may separate while stored. Briefly shake or swirl milk to mix before offering it to baby.
4. If feeding from the collection bottle, **you must use Evenflo Balance + Standard Nipples**. Other brands or types of nipples will not work effectively. When using the Evenflo Balance + Standard Nipples for feeding, squeeze nipple base and insert through the **TOP** of the cap ring. Once lip is inside cap ring, release the nipple and lightly push the inside lip towards the edge. Hand-tighten cap ring with care not to over or under tighten. *Evenflo Balance + Standard Nipples sold separately.*
5. Always test the temperature of the milk before feeding.

IMPORTANT!

- **NEVER** microwave human milk or heat directly on the stove.
- **If baby does not finish breast milk at one feeding, unfinished amount must be discarded within 2 hours.**
- Cool freshly pumped breast milk before adding to refrigerated or frozen breast milk.

Problem: PUMP WON'T TURN ON	
Possible Cause	Action
AC adapter not connected properly.	Check adapter connections (p. 9).
No batteries or low batteries.	Make sure fresh, brand-name, alkaline, non-rechargeable batteries are installed correctly (p. 9-10). If suction level 1 light is flashing when the pump is off, the batteries are too low to effectively pump.
Problem: LITTLE OR NO SUCTION	
Possible Cause	Action
Suction setting too low.	Choose a higher suction level by pressing the suction increase button (p. 12).
Valve not assembled to flange.	Attach the valve firmly to flange (p. 7).
Membrane worn, torn or not assembled to valve.	Replace membrane or ensure that it is connected properly to the valve (p. 7). To maintain optimal performance with daily use, it is recommended that membranes be replaced every 60 days.
Dried milk in pump parts.	Follow cleaning instructions. This pump can be easily disassembled and washed (p. 6 and 14).
Membrane not sealing.	Make sure valve and membrane are clean, with no debris on the surface of the seal. Make sure membrane lies flat against the surface of the valve (p. 7).
Tubing or tubing adapter is loose or not connected.	Check assembly of flange. All parts must be assembled properly and fit together snugly, without gaps, for the pump to provide maximum comfortable suction (p. 7 and 8).
Not a good seal on your breast.	Reposition the flange to get a better seal. If the breast becomes wet with milk, you may need to dry skin and flange and start again (p. 11).
Low batteries.	Make sure fresh, brand-name, alkaline, non-rechargeable batteries are installed correctly (p. 7).

Problem: MILK WON'T FLOW INTO THE BOTTLE	
Possible Cause:	Action:
The membrane is not moving on the valve.	Clean the valve and reassemble, per step 1 in Assembling the Flange section (p. 7).

Still having problems? Call our Parent's Ally Resource Center for help.

Remember, this is a personal care item for one user only. For hygienic reasons, **it CANNOT BE RETURNED TO THE STORE ONCE PACKAGE HAS BEEN OPENED.**

If you have any questions, comments, or concerns about our Evenflo breast pumps, please contact our Parent's Ally Resource Center.

This product is designed for use by one person. Use of this product by more than one person will void the warranty.

From the original purchase date of this Product, for a period of three years for the Product's motor and 90 days for all other Product components and accessories, Evenflo provides a warranty to the original end user ("Purchaser") against defects in material or workmanship. Evenflo's sole obligation under this express limited warranty shall be, at Evenflo's option, to repair or replace any Product, component or accessory that is determined to be defective by Evenflo and determined to be covered by this warranty.

Repair or replacement as provided under this warranty is the exclusive remedy of the Purchaser. For full warranty replacement options to be available, you must have proof of at least one of the following: (1) a purchase receipt, (2) proof of delivery through insurance, or (3) prior registration of your pump at www.registermypump.com. Refunds will only be available with a purchase receipt dated in the last 6 months. This express limited warranty is extended by Evenflo ONLY to the original Purchaser of the Product and is not assignable or transferable to subsequent purchasers or end users of the Product. For warranty service, contact Evenflo's Parent's Ally Resource Center at 1-855-EFI-BABY (334-2229) or www.evenflofeeding.com.

EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THIS PRODUCT IS HEREBY DISCLAIMED. EVENFLO SHALL NOT BE LIABLE FOR ANY INCIDENTAL, CONSEQUENTIAL, SPECIAL OR PUNITIVE DAMAGES OF ANY KIND FOR BREACH OF THE EXPRESS LIMITED WARRANTY ON THIS PRODUCT, OR ANY WARRANTY IMPLIED BY OPERATION OF LAW. OTHER THAN THE EXPRESS LIMITED WARRANTY SET FORTH ABOVE, THERE ARE NO OTHER WARRANTIES THAT ACCOMPANY THIS PRODUCT AND ANY ORAL, WRITTEN OR ANY OTHER EXPRESS REPRESENTATION OF ANY KIND IS HEREBY DISCLAIMED.

Parent's Ally

USA/Canada: 1-855-EFI-BABY (334-2229); www.evenflofeeding.com
 Manufactured and Distributed by Evenflo Feeding, Inc.

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Fabricado y distribuido por Evenflo Feeding, Inc.