
**inogen connect**  
GETTING STARTED GUIDE

## Before You Start

### 1 Download App

From your smart device, go to the **APP STORE (APPLE)** or **Google Play (Android)**, search for the **Inogen Connect App** and download the software.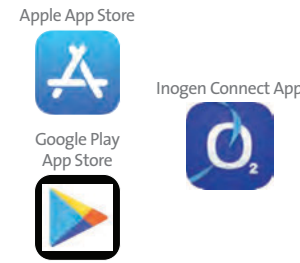

### 2 Plug In Your Inogen One G5

Connect the AC power supply cord to the concentrator. Plug the power supply AC cord into an electrical outlet. **Do not power on G5 concentrator.**

inogen connect

## Pairing

### 3 Pair Your Mobile Device With Your G5

- Go to **Settings** on your smart device and **turn on Bluetooth**.
- Slide the button to the right to turn on Bluetooth.
- Next, **turn on Bluetooth on your concentrator** by holding down the minus button until the Bluetooth symbol appears on the concentrator's display screen.

### 4 Open Inogen Connect App


The Inogen Connect Code can be found in your confirmation email or provided by your home care provider. Enter your Inogen Connect Code by:

- Manually entering.
- Use your smart device's camera to scan the QR code found on your invoice (must allow access to your camera through the app)

*Pairing is the term used to describe the process of wirelessly connecting two Bluetooth devices for the first time. "Pairing" allows the devices to recognize each other and creates a unique lasting connection between the specific devices.*

Example QR code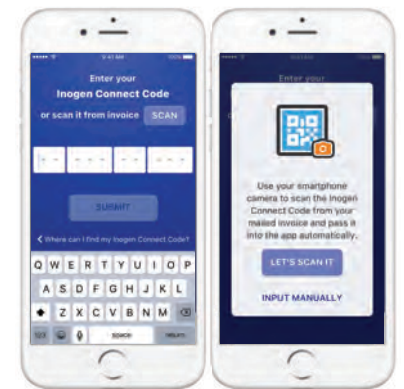


inogen connect

## Pairing


### 5 Search for Concentrator

Click on "Search for Concentrator" at the bottom of your screen.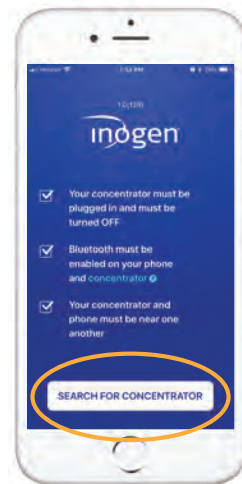

### 6 Select Serial #


When device is found, select your desired **Concentrator Serial Number** on your smart device.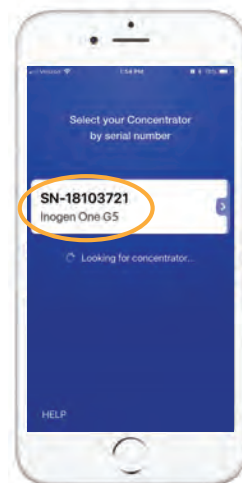

### 7 Terms & Conditions


Read the **Terms and Conditions** and if you choose to accept, select the "I Accept" button at the bottom of your screen.

There may be an additional consent statement for some locations. Read the **consent statement** and if you choose to accept, select the "I Accept" button at the bottom of your screen.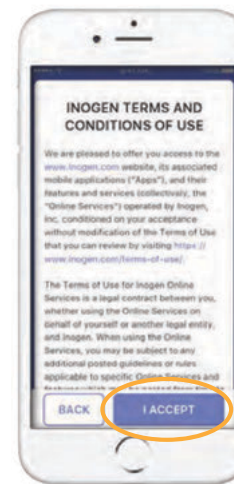

### 8 Press Bell Button


Press and hold the **bell button** on your G5 concentrator to finish pairing.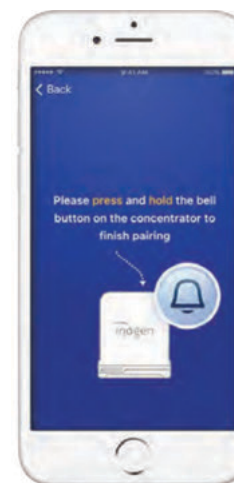

inogen connect

### 9 Wait to Pair

Please wait - your concentrator is pairing.

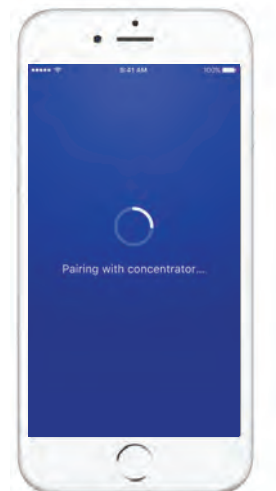### 10 Pairing Complete

Your **Inogen One G5** is now paired with your smart device! Now you can turn on your Inogen One G5 and use it normally. The information displayed on your **Inogen Connect** screen will vary depending on your concentrator's current state.

For further information, see the Inogen Connect **FAQ** section at [www.inogen.com/app/](http://www.inogen.com/app/)

**THE PURPOSE OF THE INOGEN MOBILE APP IS TO PROVIDE ADDITIONAL CONVENIENCE TO THE PATIENT. HOWEVER, THE USER INTERFACE PANEL REMAINS THE PRIMARY SOURCE OF INFORMATION TO WHICH THE PATIENT SHOULD REFER.**

**CE 2797**  
**EC REP**  
Inogen, Inc.  
301 Coromar Drive,  
Goleta, CA 93117  
1-877-4-INOGEN (Toll Free)  
1-877-446-6436 (Toll Free)  
1-805-562-0515 (International)  
info@inogen.net  
www.inogen.net  
Europe Authorized Representative  
EMERGO EUROPE  
Prinsessegracht 20  
2514 AP, The Hague  
The Netherlands

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