
AeroEclipse II

Breath Actuated Nebulizer (BAN)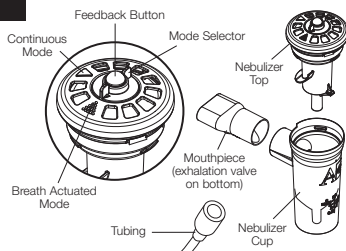

Instructions for Use

1. Ensure mouthpiece is inserted into side opening of the nebulizer with the exhalation valve facing down.
2. Prior to use, inhale and exhale through the nebulizer mouthpiece to verify the movement of the green feedback button located on top of the device.
3. Unscrew and remove top of nebulizer. Place prescribed medication into nebulizer cup (maximum cup volume 6 mL). Reattach top and gently hand tighten.
4. Attach one end of the supplied tubing to the fitting in the bottom of the nebulizer and the other end to the flowmeter or compressor. Ensure both ends are securely engaged.
5. Set the flowmeter to 7 to 8 litres per minute (Lpm) or turn on compressor capable of delivering a minimum of 15 to 50 P.S.I. and a flow of 2.75 to 8 Lpm.
6. Place mouthpiece in mouth, and inhale slowly and deeply. As you inhale, the green feedback button on top of the nebulizer will move into the fully down position, indicating that the **AeroEclipse II BAN** is producing aerosol in response to inhalation. Exhale normally. A valve on the mouthpiece opens allowing passive exhalation, as the green feedback button returns to the up position indicating no aerosol is being produced. Do not place the lips over the exhalation valve on the bottom of the mouthpiece. Doing so will prevent the valve from functioning properly.
7. Continue to inhale from the nebulizer until nebulizer begins to sputter, or as instructed by your physician or qualified healthcare worker.

The **AeroEclipse II BAN** can be operated in a Continuous Mode by rotating the mode selector 90

degrees clockwise to the continuous mode position (Δ). The nebulizer can be returned to the Breath Actuated Mode by rotating the mode selector 90 degrees counterclockwise to the breath actuated position (Δ).

Cleaning Instructions

Rotate the mode selector to the breath actuated position (green feedback button is up, mode selector has rotated counterclockwise) for both the washing and drying cycles. Between treatments, remove the supplied tubing from the bottom of the nebulizer, unscrew and remove top of nebulizer and wash nebulizer parts in a warm water and liquid dish detergent solution. Rinse in clean warm water and allow parts to dry thoroughly before reassembling.

Single Patient Use

The device should be replaced after approximately 7 days or immediately if components are cracked or the nebulizer does not function properly. This device is not intended to be reprocessed and should not be autoclaved or boiled.

CEN 13544-1 Data

The Aerosol Output Rate for the **AeroEclipse II BAN** is 0.16 ± 0.01 mL/min and the Aerosol Output is 0.51 ± 0.03 mL in Breath Actuated Mode.

Indications for Use

The **AeroEclipse II BAN** is intended to be used by patients who are under the care or treatment of a licensed healthcare provider. The device is intended to be used by these patients to administer aerosolized medication prescribed by a physician or healthcare provider. The intended environments for use include the home, hospitals and clinics.

CAUTIONS:

- Supervision is necessary when this device is used for treating children.
- The **AeroEclipse II** Breath Actuated Nebulizer can deliver aerosolized medication in either a Breath Actuated or Continuous Mode. In Breath Actuated Mode, medication is delivered on inspiration only and may result in an increase of the dosage to the lungs. Healthcare providers should take this information into consideration with regards to dosage.
- Failure to properly follow the above cleaning instructions may have an adverse affect on device performance.

REORDER NUMBER

AeroEclipse II BAN 10550191050

725 Third Street
London, Ontario, Canada, N5V 5G4
customerservice@trudellmed.com
+1-519-455-7060 +1-519-455-7858
www.trudellmed.com

Trudell Medical International*

Canadian Hospital Distribution:
Trudell Medical Marketing Limited
758 Third Street, London, Ontario, Canada N5V 5J7
tmm@tmm.com 1-800-265-5494
www.tmm.com

Authorized European Representative:
Trudell Medical International Europe Limited
BioCity Nottingham, Pennyfoot Street,
Nottingham, United Kingdom NG1 1GF
customerservice@trudellmed.com

10516-001 Rev E. Copyright © Trudell Medical International 2007, 2009. All rights reserved. * are trade marks and registered trade marks of Trudell Medical International.

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10516-001 Rev E. Copyright © Trudell Medical International 2007, 2009. Todos los derechos reservados. * son marcas comerciales y marcas comerciales registradas de Trudell Medical International.