OMBRA*

Table of Contents	1	EN
Table des matières	10	FR
Índice	20	ES

TABLE OF CONTENTS

Package Contents 2

Indications for Use 5

Important Safety Information 5

Electrical Safety 5

Before You Begin 6

Cleaning and Maintenance 8

 Air Filter 8

 Nebulizer 8

EZ Twist Tubing 8

 Compressor 8

Storage 8

Disposal 9

Technical Specifications 9

-  Class II
-  Indoor use only
-  Disposal of Electrical & Electronic Equipment (WEEE)
-  Consult instructions for use
-  Type BF Applied Part (mouthpiece)

 Medical Equipment Compressor Nebulizer with Respect to Electrical Shock Fire and Mechanical Hazards
 Only in Accordance with UL 60601-1 and CAN/CSA C22.2 No. 601.1


PACKAGE CONTENTS

This package contains the following items:


- ❑ **Ombra*** Table Top Compressor

- ❑ **Ombra*** Table Top Compressor Owner's Manual (this guide)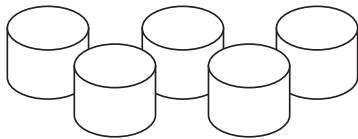


- ❑ Five replacement Air Filters

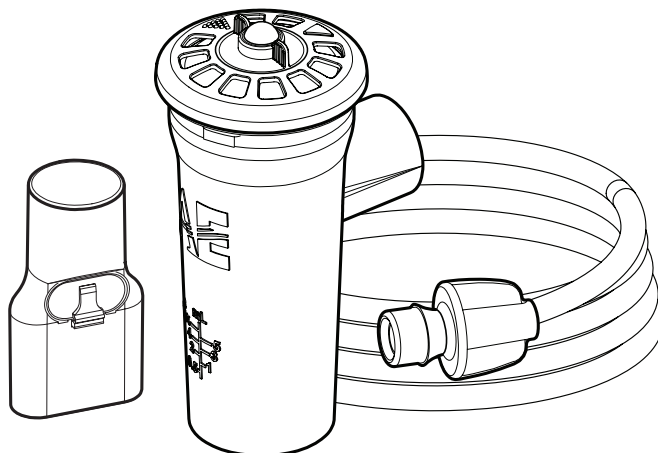- ❑ Reusable **AEROECLIPSE*** XL Breath Actuated Nebulizer (package includes an owner's manual, two nebulizers with mouthpiece and two **EZ Twist Tubes**)

FIGURE 1

Top View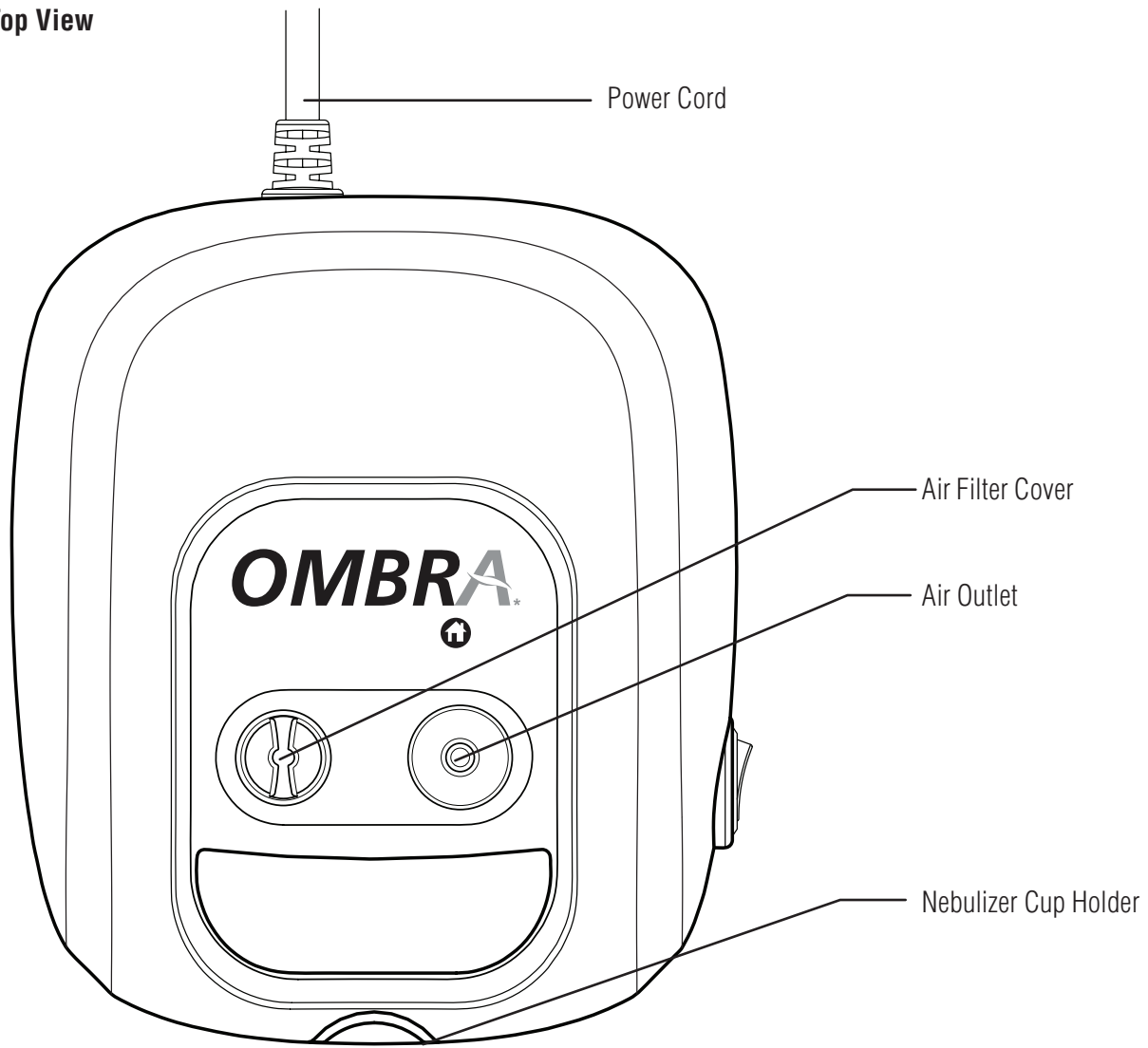

Side View


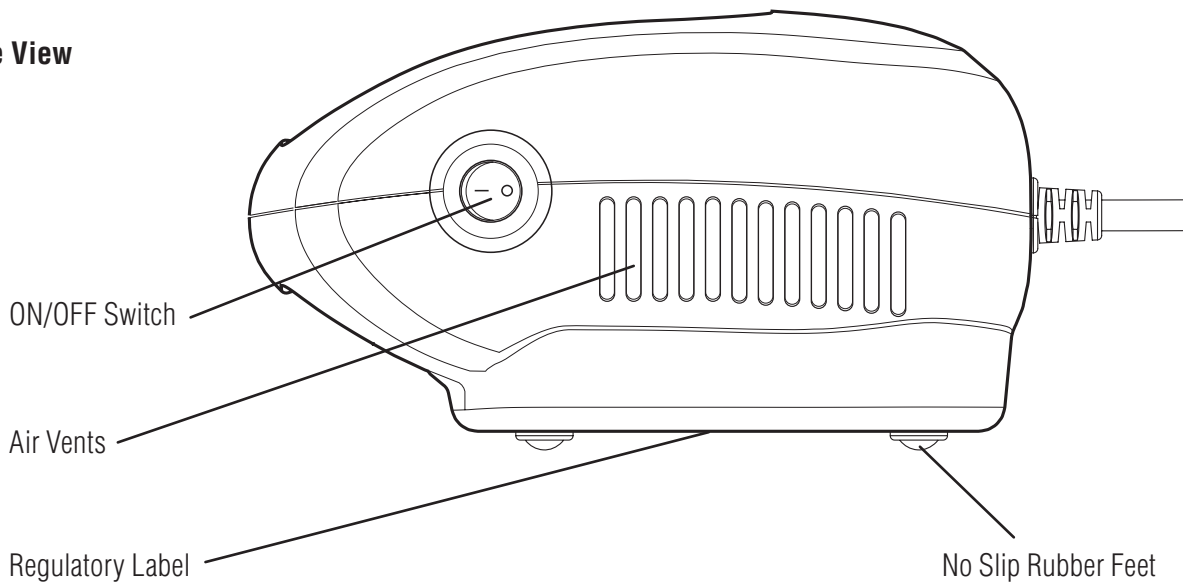
FIGURE 2

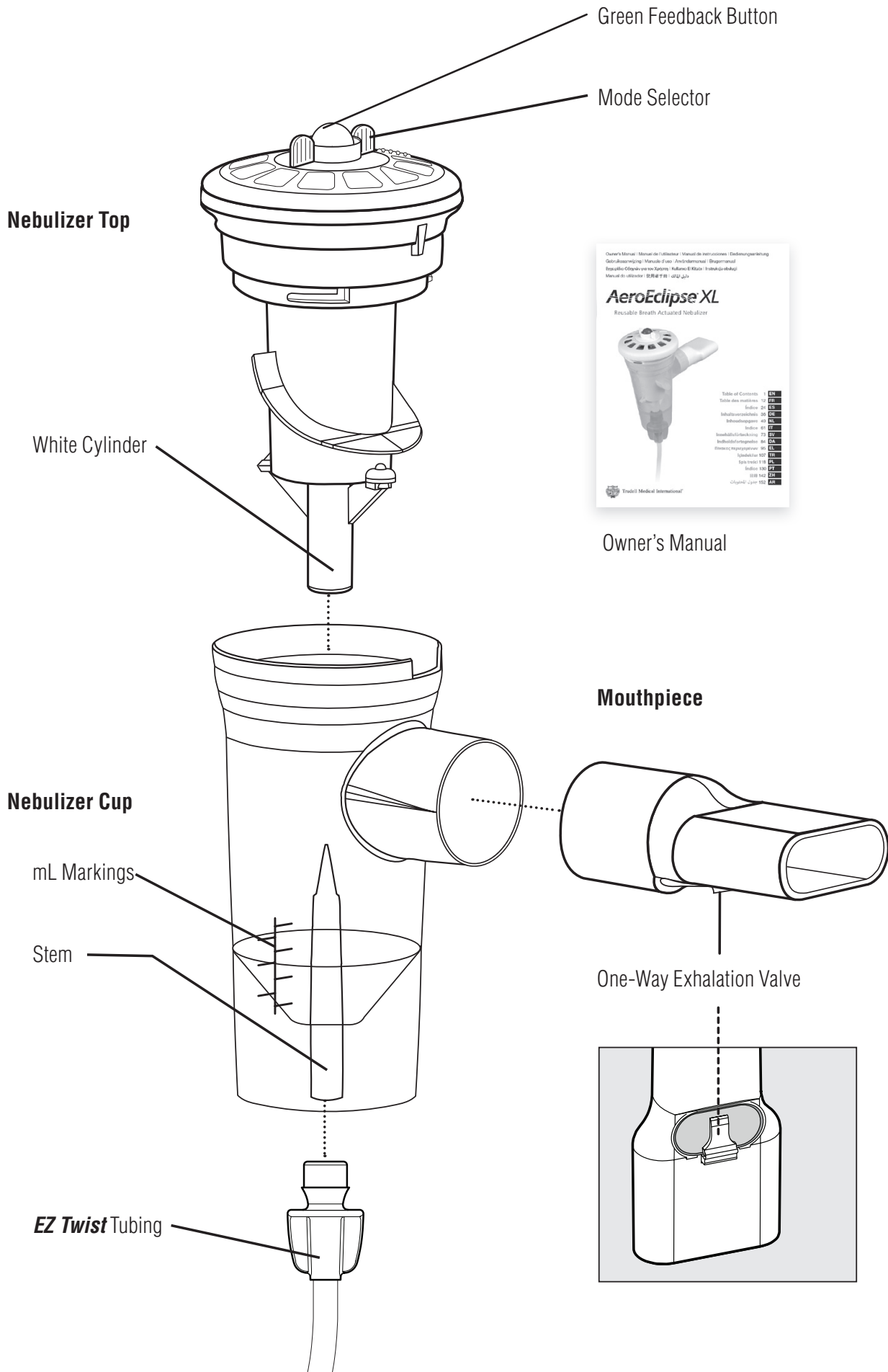INDICATIONS FOR USE

The **Ombra*** Table Top Compressor is designed to deliver compressed air to a jet or pneumatic nebulizer. This compressor may be used with adult, child or paediatric patients. The **Ombra*** Table Top Compressor is a medical device, and should only be used as directed by your healthcare professional. The intended environments for use include the home, hospitals and clinics.

The compressor has been designed for use with the reusable **AEROECLIPSE*** XL Breath Actuated Nebulizer (BAN). For use with other nebulizers, please ensure you refer to the Technical Specification Table to ensure compatibility. If you are unsure if your nebulizer is compatible with the **Ombra*** Table Top Compressor, please consult with your healthcare professional before use as it could affect the effectiveness of your treatment.

Be sure to read all of the instructions before using your new **Ombra*** Table Top Compressor. These instructions should be stored in a safe place for future reference as this booklet contains important information about the proper functioning of your compressor. Failure to observe these instructions may result in injury and/or damage to the compressor.

IMPORTANT SAFETY INFORMATION

The following statements will be used to draw attention to potential use errors that could cause harm.

- ① **NOTE** - Indicates information to which the user should pay special attention.
- ⚠ **CAUTION** - Information for preventing damage to the product.
- ⚠ **WARNING** - Important safety information for hazards that could cause serious injury.
- ⚠ **DANGER** - Urgent safety information for hazards that will cause serious injury or death.

ELECTRICAL SAFETY

As with any electrical device, extra precautions should be taken to prevent injury or death. Special attention should be paid when using electrical appliances around young children. Always unplug the compressor when not in use.

⚠ **DANGER:** To reduce the risk of serious injury or death from electrocution:

- Do not use or store this device where it can fall or be pulled into water or other liquid, such as a bathtub or sink. Do not use while bathing.
- If the compressor falls into water or other liquid, do not reach for it. Unplug the compressor immediately.
- If the power cord appears damaged or frayed, the compressor should not be used and should be unplugged immediately.

⚠ **WARNING:** To prevent serious injury, fire or burns and to reduce the risk of damage to the product:

1. Ensure you are using the correct compressor for the voltage and current available. This information will be printed on the label on the bottom of your compressor.

120 Volt Compressor

The polarized plug will only fit fully into the electrical outlet when it is positioned the correct way. If you are having difficulty plugging the compressor in, reversing the plug may resolve this.

2. The **Ombra*** Table Top Compressor should be unplugged when not in use. Do not plug the compressor into an electrical outlet that makes it hard to disconnect from the power source or where the outlet is hard to access. The AC power plug is served to disconnect the device.
3. Never operate the compressor if:
 - a. It appears to have damage to the power cord, power cord connection or plug;
 - b. It has been dropped or damaged;
 - c. It has been exposed or dropped into water or other liquids;
 - d. It does not appear to be operating properly.

If any of these occur, return the compressor to your **Ombra*** Authorized Distributor for replacement.

4. Keep any electrical cords away from sources of heat and avoid overloading electrical outlets.
5. Never use the compressor when sleeping or drowsy.
6. Do not operate or store the compressor where the **AIR VENTS** may be obstructed as this may cause the compressor to overheat. The **Ombra*** Table Top Compressor has a safety feature that will automatically shut off the compressor if it overheats. If this occurs, wait 30 minutes before turning the compressor back on.
7. The compressor surfaces may become hot during prolonged treatments. Allow surfaces to cool before handling to avoid contact burns.
8. Compressor should be used on a clean, flat surface. Do not use or store the compressor in areas where there is excessive dust or particulate matter in the air. Compressor should only be used indoors.
9. This compressor should not be used near where oxygen is being administered or in the presence of any flammable material and/or gas.

10. Never drop or insert any object into any openings.
11. Ensure the compressor is unplugged prior to cleaning.
12. The **Ombra*** Table Top Compressor could cause interference with some types of electronic equipment. To reduce the risk of interference, move it further away from the electrical equipment.
13. Use this compressor only for its intended use as described in this guide. Do not open or disassemble the **Ombra*** Table Top Compressor. Refer all servicing to your **Ombra*** Authorized Distributor.
14. Do not use attachments not recommended by the manufacturer.

⚠ **WARNING:** Close supervision is necessary when this product is used by or near infants, children or anyone requiring assistance to use the device. It contains parts that may present a choking or strangulation hazard. **DO NOT LEAVE CHILDREN UNATTENDED DURING TREATMENTS.**


BEFORE YOU BEGIN

For compressor part descriptions see Figure 1.


Ensure you have thoroughly read and understood all of the instructions for use for your **Ombra*** Table Top Compressor before you begin your treatment. Keep your **OWNER'S MANUAL** in a safe place for future reference. To download an electronic copy of this **OWNER'S MANUAL**, visit www.aeroeclipse.com

1. Place the compressor on a clean, flat surface or table top. Ensure there is nothing blocking the **AIR VENTS** on either side of the compressor. Verify that the **ON/OFF SWITCH** is in the 'Off' position.

FIGURE 3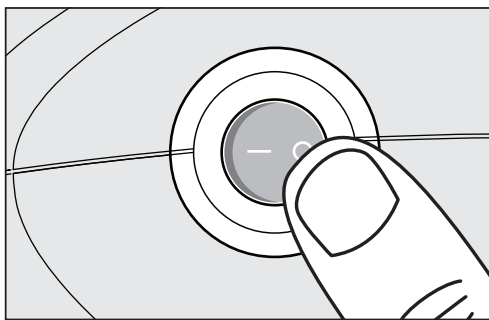

OFF

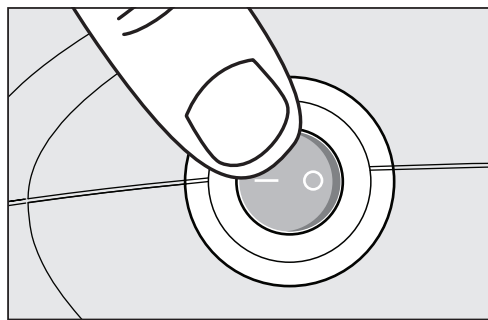ON

2. Read and understand the instructions included with your reusable **AEROECLIPSE*** XL Breath Actuated Nebulizer (BAN).
3. Assemble the nebulizer and ensure you have installed the mouthpiece with the **EXHALATION VALVE** pointing down. (Fig. 2). If you are using your reusable **AEROECLIPSE*** XL BAN with a **ComfortSeal*** Mask (Fig. 4), remove the mouthpiece and insert the elbow adapter directly into the nebulizer cup as shown in Figure 5.

FIGURE 4



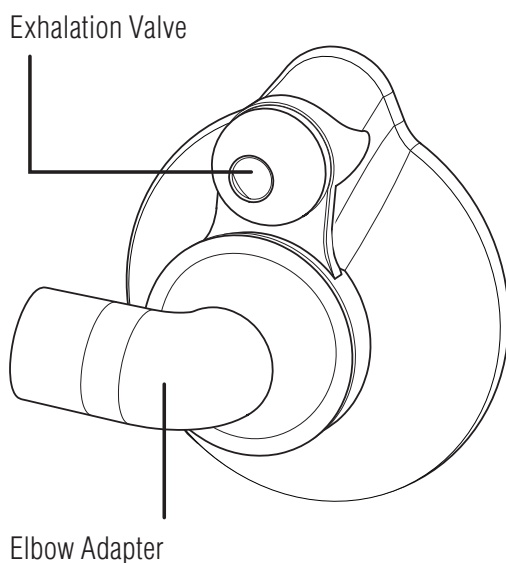
FIGURE 5

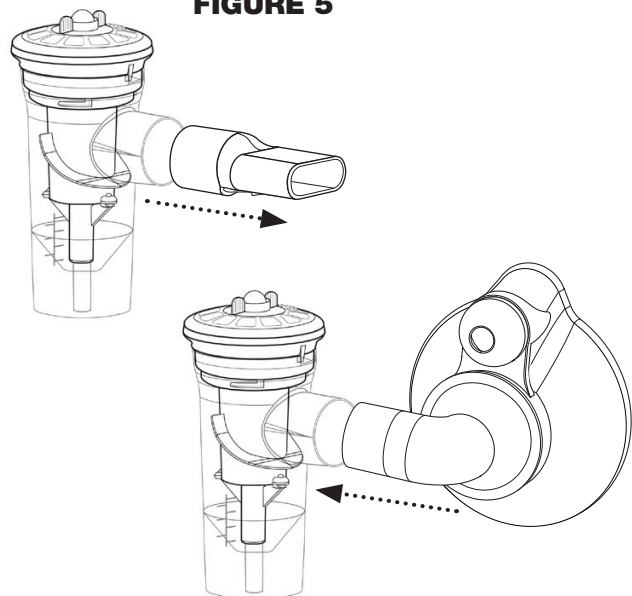
4. Unscrew and remove the **NEBULIZER TOP**. Carefully place prescribed medication into **NEBULIZER CUP**. Reattach the **NEBULIZER TOP**, making sure to line up the **WHITE CYLINDER** with the **STEM** of the **NEBULIZER CUP** and gently hand tighten until a stop is detected. Correct assembly is shown in Figure 6. Ensure nebulizer is kept in an upright position so medication does not spill.

FIGURE 6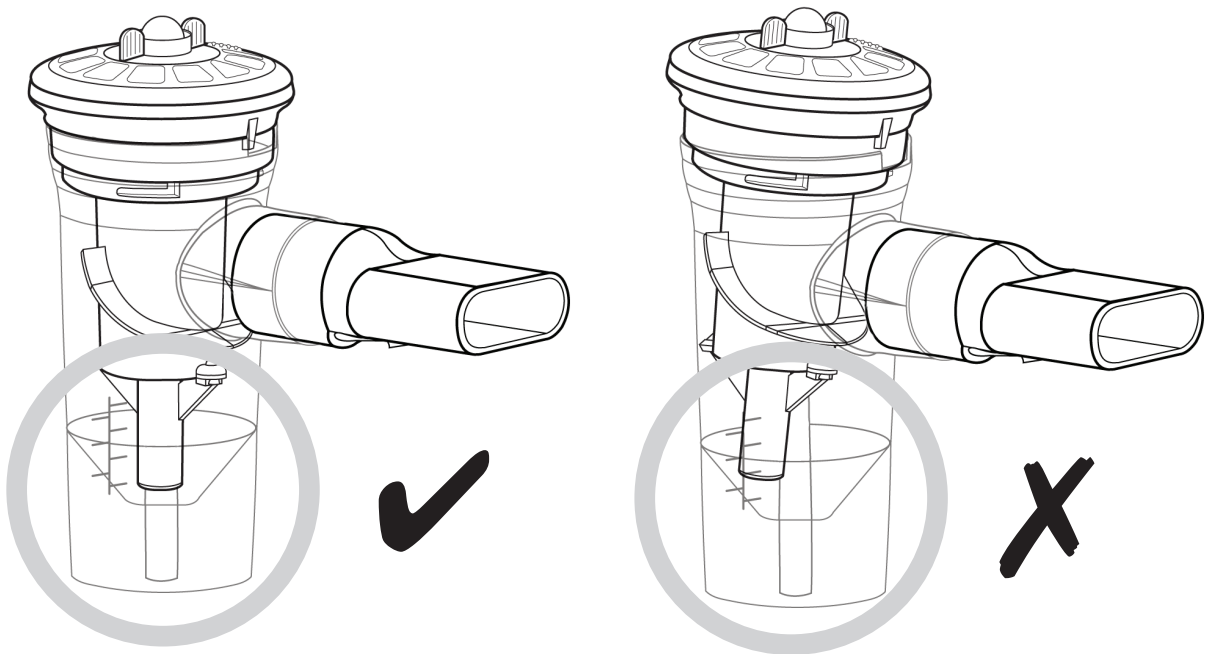

5. Connect one end of the **EZ Twist TUBING** to the fitting on the bottom of the nebulizer and the other end to the **AIR OUTLET** on the compressor. Ensure both ends are securely engaged. The nebulizer may then be placed in the **CUP HOLDER** on the compressor until you are ready to start your treatment as shown in Figure 7.

FIGURE 7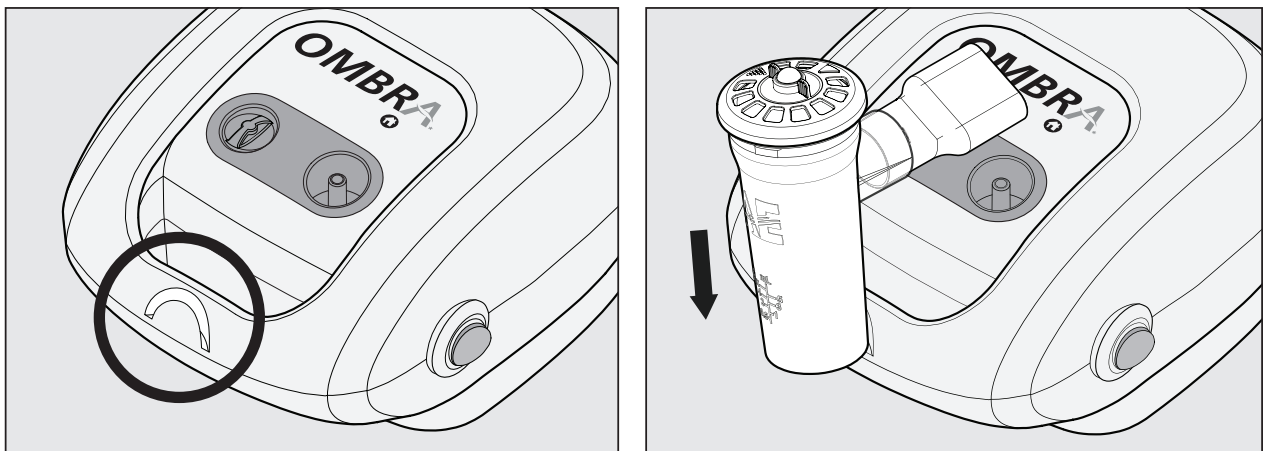

6. Plug the **Ombra*** Table Top Compressor into an electrical outlet.
7. In a relaxed, comfortably seated position, pick the nebulizer up out of the **CUP HOLDER**. Switch on the compressor and start your treatment.
8. When your treatment is complete or during a rest in your treatment, the nebulizer may be placed back in the **CUP HOLDER** and the compressor switched off.

① **NOTE:** The compressor should be unplugged when not in use.

CLEANING AND MAINTENANCE


AIR FILTER

Your **Ombra*** Table Top Compressor is equipped with a special air filter that will slowly change colour with use. The original filter is bright white, but it will slowly turn grey with use. Your air filter should be changed every 6 months, or more often as visual inspection indicates. The compressor should never be used without an air filter installed.

To change the filter (Figure 8)

1. Remove **AIR FILTER COVER** by holding the cover and turning counterclockwise approximately 90 degrees. Pull the cap and filter cover out.
2. Remove filter from the cover, and dispose of the old filter.
3. Insert a new filter into the cover, then place the **AIR FILTER COVER** back into the compressor and turn clockwise approximately 90 degrees to tighten.
4. Do not over tighten. There will be resistance when the **AIR FILTER COVER** is securely fastened.

FIGURE 8

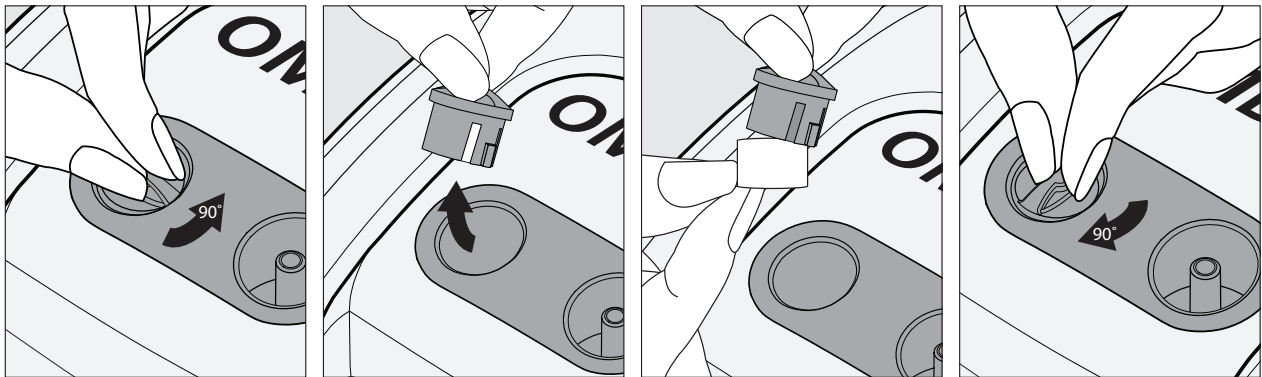⚠ CAUTION: Do not wash or reuse the filter. Use only **Ombra*** Replacement Filters (Part Number: 105544-001). Do not attempt to substitute any other material to act as a filter as this will result in damage to the compressor and may void your warranty.

NEBULIZER

Refer to the **OWNER'S MANUAL** included with your reusable **AEROECLIPSE*** XL BAN for instructions on how to properly clean your nebulizer.

EZ TWIST TUBING

Occasionally you may notice condensation on the inside of your oxygen tubing. This is normal. To remove the condensation, simply connect one end of the **EZ Twist** TUBING to the **AIR OUTLET** on the compressor and allow the compressor to run without the nebulizer attached. This will force air through the tubing, drying the inside. To clean the exterior, wipe with a damp cloth. Ensure the tubing is dry prior to storage.

COMPRESSOR

Prior to cleaning the compressor, ensure the unit is unplugged from the electrical outlet. Use a damp cloth to wipe the exterior of the compressor housing. Never use liquid or spray cleaners directly on the compressor.

⚠ CAUTION: Liquids can cause damage to the electrical components inside your compressor and lead to malfunction. In the event any liquid enters the unit, please contact your **Ombra*** Authorized Distributor. Do not use the compressor until it has been inspected, even if it appears to be functioning properly.

STORAGE

The **Ombra*** Table Top Compressor and reusable **AEROECLIPSE*** XL BAN should always be stored together with all of the parts in a dry, dust-free environment.

⚠ DANGER: Do not attempt to disassemble the compressor housing as this presents an electric shock hazard. There are no parts inside the compressor housing that require maintenance or replacement. Should you require repair or service for your compressor, please contact your **Ombra*** Authorized Distributor.

⚠ WARNING: No modification of this equipment is allowed.

DISPOSAL

This product falls within the scope of Directive 2002/96/EC of the European Parliament and the European Council of January 27, 2003 on waste electrical and electronic equipment and is included in product category 8: Medical devices for the purposes thereof. Accordingly, this product must not be disposed of with domestic waste. The disposal regulations prevailing in the respective member countries must be observed (e.g. disposal by local authorities or dealers).

TECHNICAL SPECIFICATIONS

See label on bottom of compressor to verify your compressor voltage.

120 Volt Compressor	
Model Number	105510
Weight	3.5 lbs (1.59 kg)
Dimensions	7" x 6.1" x 3.9" (17.8 cm x 15.5 cm x 10 cm)
Sound Level	60 dB(A)
Electrical Requirements	120 Volt
Power cord length	5 feet (1.52 metres)
Power	132VA
Frequency	60 Hz
Duty Cycle	30 Minutes On/30 Minutes Off
Fuse Rating	5A
Operating Temperature Range	+15°C to +40°C (59°F to 104°F)
Operating Humidity	15 to 95% (non-condensing)
Storage/ Transport Conditions	-20°C to +60°C (-4°F to 140°F)
Storage/Transport Humidity	15 to 95% (non-condensing)
Maximum Pressure	37.0 psi (2.55 bar)
Free Flow Air Rate	9.5 L/min
Supplied Reusable Nebulizer Operating Pressure	19.5 psi (1.34 bar)
Supplied Reusable Nebulizer Operating Liter Flow	4.0 L/min
Recommended Replacement Period	3 years
Information Last Updated	January 2012

REORDER NUMBERS

Reusable AEROECLIPSE* XL BAN (Includes: Reusable AEROECLIPSE* XL BAN with mouthpiece, EZ Twist Tubing, owner's manual)	10551393010
Reusable ComfortSeal* Mask with Elbow Adapter, Small (0 – 18 months)	10550393010
Reusable ComfortSeal* Mask with Elbow Adapter, Medium (1 – 5 years)	10550293010
Reusable ComfortSeal* Mask with Elbow Adapter, Large (5+ years)	10550493010
Air Filters, (5 pk)	105544-001

To find the **Ombra*** Authorized Distributor in your country, visit www.aeroeclipse.com

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Trudell Medical International

725 Third Street
London, Ontario, Canada, N5V 5G4
Phone: +1-519-455-7060
Fax: +1-519-455-7858
Email: customerservice@trudellmed.com
Website: www.trudellmed.com

Distributed By:

Canadian Hospital Distributor:
Trudell Medical Marketing Limited
758 Third Street
London, Ontario, Canada N5V 5J7
Email: tmml@tmml.com
Website: www.tmml.com

Trudell Medical International Europe Ltd.
BioCity Nottingham, Pennyfoot Street
Nottingham, NG1 1GF, UK
Email: customerservice@trudellmed.com
Website: www.trudellmed.com

105510

105544