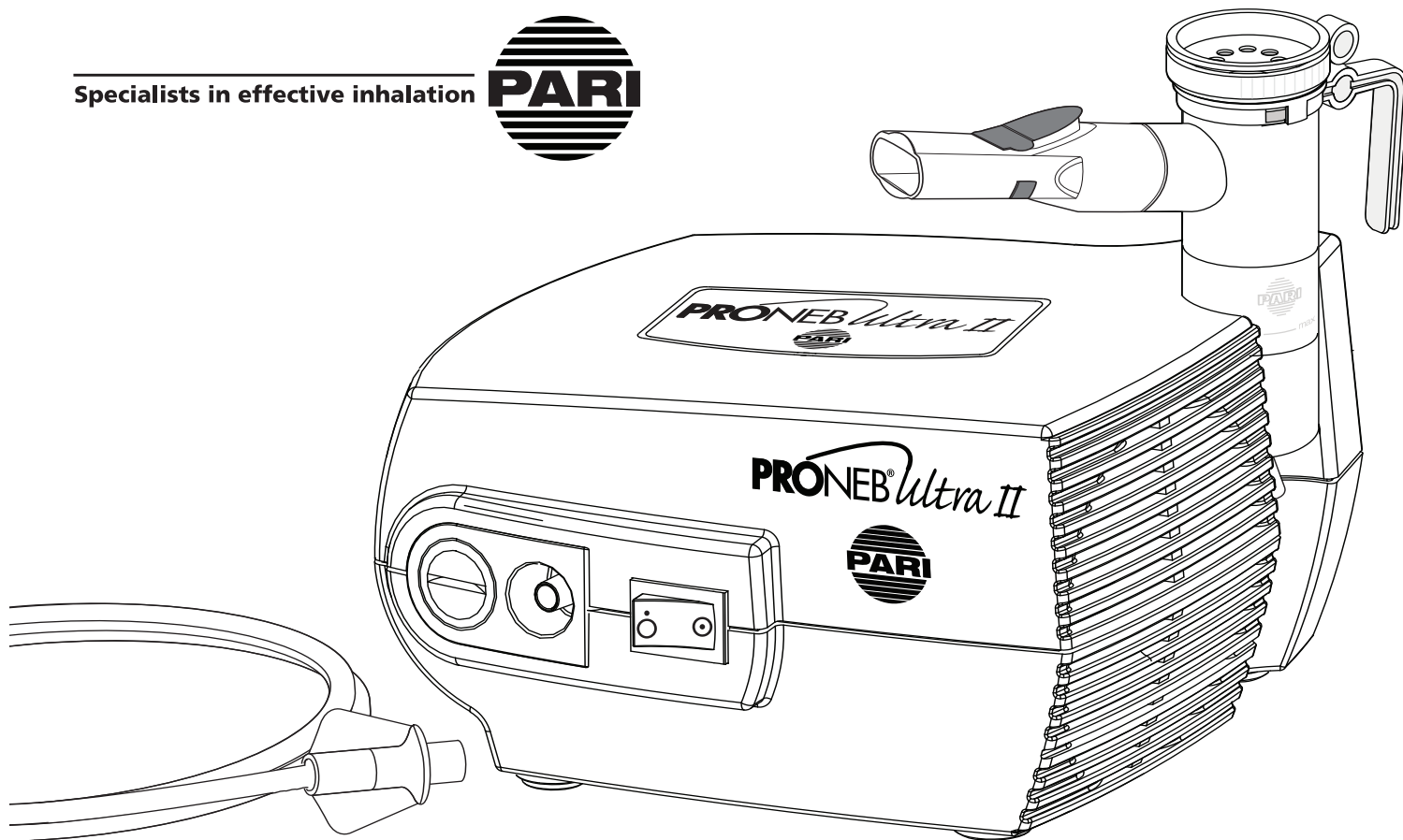
Specialists in effective inhalation

## INSTRUCTIONS FOR USE

Read instructions and safety precautions before use.

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**PRONEB<sup>®</sup>** *Ultra II*  
COMPRESSOR NEBULIZER SYSTEM

## Table of Contents

I. Introduction	3
II. Features and Functions	3
III. Safety Precautions	3
IV. Getting Started	5
V. Set-Up	6
VI. Specifications	6
VII. Maintenance of the <b>PRONEB</b> ® ULTRA II Compressor	10
VIII. <b>PRONEB</b> ® ULTRA II Compressor Limited Warranty	11
IX. Repair Return Procedure	12

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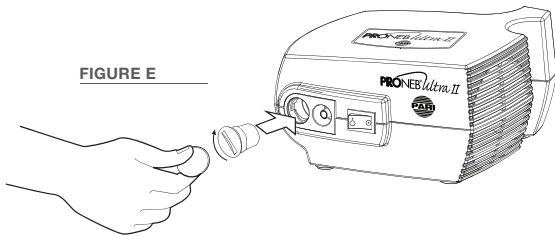
FIGURE F

## I. Introduction

These Instructions For Use contain Information and Safety Precautions on the **PRONEB**<sup>®</sup> ULTRA II Compressor Nebulizer System. The **PRONEB**<sup>®</sup> ULTRA II compressor is a AC-powered air compressor intended to provide a source of compressed air for use with jet nebulizers. The **PRONEB**<sup>®</sup> ULTRA II compressor is intended for adult and pediatric patients. Open the inside front cover to view illustrations for setup of your **PRONEB**<sup>®</sup> ULTRA II Compressor. The illustrations will be referenced throughout these Instructions For Use. Separate Instructions For Use are included with the enclosed PARI Reusable Nebulizer.

Before using your **PRONEB**<sup>®</sup> ULTRA II Compressor, you should read and understand the entire Instructions For Use for both the **PRONEB**<sup>®</sup> ULTRA II Compressor, and the PARI Reusable Nebulizer. Take special note of all safety precautions marked **DANGER** and **WARNING**. Save these Instructions For Use for future reference.

## II. Features And Functions (see illustrations, page 2)

1. **PRONEB**<sup>®</sup> ULTRA II Compressor
2. Air Filter
3. Air Outlet
4. On/Off Switch
5. Reusable Nebulizer Holder
6. Wing Tip™ Tubing
7. a. PARI LC<sup>®</sup> Plus or PARI LC<sup>®</sup> Star Reusable Nebulizer  
b. PARI LC<sup>®</sup> Sprint Reusable Nebulizer
8. Power Cord

## III. Safety Precautions

IMPORTANT SAFEGUARDS

READ ALL INSTRUCTIONS BEFORE USING.

 **WARNING**

CAUTION: U.S. Federal Law restricts this device to sale by or on the order of a physician. Use only when prescribed by a physician and with the prescribed medication.

Read all warnings and instructions before use. Use the product only for its intended use as described in these Instructions For Use. Improper use can result in serious or fatal illness/injury, improper treatment or property damage.

Use only with nebulizers and tubing authorized by PARI for this device. Other nebulizers and tubing may result in improper treatment.

When using electrical and medical products, especially when children are present, basic safety precautions should always be followed, including the following:

 **DANGER**

To reduce the risk of serious or fatal injury from electrocution:

1. Do not place or store product where it can fall or be pulled into a bathtub, sink, water, or other liquid. Do not place in or drop into water or other liquid. Do not use while bathing.
2. Do not reach for a product that has fallen into water or other liquid. Unplug immediately.
3. Always unplug this product immediately after using.

 **WARNING**

To reduce the risk of serious or fatal injury from electrocution, fire or burns and to reduce the risk of damage and malfunction to the unit:

1. Before use, check the serial number label on the bottom of the **PRONEB**<sup>®</sup> ULTRA II Compressor to ensure that the voltage and current indicated on the unit correspond to the voltage and current available.

2. Never operate this product if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged, or if it has been exposed to any liquids inside the case. Return the product to an authorized PARI Service Center for examination and repair.

3. Keep all electrical cords away from heated surfaces. Do not overload wall outlets or use extension cords.

4. This appliance has a polarized plug (one blade is wider than the other). As a safety feature, this plug will fit in a polarized outlet only one way. If the plug does not fit fully in the outlet, reverse the plug.

5. Never block the air vents of this product or place it on a soft surface, such as a bed or couch, where the air vents may be blocked. Keep the air vents free of lint, hair, and similar substances. Blockage could cause excessive heat buildup. A product should never be left unattended when plugged in.

6. Do not operate outdoors.

7. Do not operate where oxygen is being administered in a closed environment such as an oxygen tent.

8. Never drop or insert any object into any openings.

9. Be sure the **PRONEB**<sup>®</sup> ULTRA II Compressor is unplugged prior to cleaning.

10. Do not open or disassemble the **PRONEB**<sup>®</sup> ULTRA II Compressor. Refer all servicing to an authorized PARI Service Center.

11. This device needs special precautions regarding EMC and must be installed and put into service according to the EMC information provided in these Instructions For Use.

12. Portable and mobile RF communication devices can disrupt electrical medical equipment. Technical electromagnetic compatibility data is available in Section VI of these Instructions For Use.

13. Use only with accessories, transducers, and cables authorized by PARI for this device. Use of this device with unauthorized accessories, transducers, and cables may result in increased electromagnetic emissions or decreased electromagnetic immunity of the device.

14. This device should not be used adjacent to, or stacked with other equipment. If adjacent or stacked use is necessary, this device should be observed to verify normal operation.

15. Not suitable for use in the presence of flammable anesthetic mixture with air or oxygen or nitrous oxide.

16. When the switch is in the off position and the compressor is not running there will still be electrical potential inside the unit. To remove power from the compressor the unit must be unplugged from the wall.

## **WARNING**

Close supervision is necessary when this product is used by or near the physically or mentally impaired or children. Never use while sleeping or drowsy. This product contains small parts that may present a choking hazard to small children. The tubing also presents a strangulation hazard. Always use close adult supervision when administering a treatment to a child. **DO NOT LEAVE CHILD UNATTENDED DURING TREATMENTS.**

### **SAVE THESE INSTRUCTIONS**

## **IV. Getting Started**

1. Check the **PRONEB**<sup>®</sup> ULTRA II Compressor packing box and make sure the following basic components are included:

- + **PRONEB**<sup>®</sup> ULTRA II Compressor
- + Air Filter (in compressor) (reorder #041E4851P12)
- + **PRONEB**<sup>®</sup> ULTRA II Compressor Instructions For Use
- + PARI Reusable Nebulizer Set with Instructions For Use

If you did not receive the components listed, please contact the dealer where you purchased your **PRONEB**<sup>®</sup> ULTRA II Compressor. Some models may include additional accessories such as an instructional DVD, carrying case, disposable nebulizer, etc.

2. It is a good idea to always keep an extra PARI LC® Reusable Nebulizer and extra compressor air filter on hand at all times. You may need it if parts become lost or damaged.

## V. Set-Up

1. Place the **PRONEB**® ULTRA II Compressor on a stable, flat surface. Do not place on cushioned, upholstered, or carpeted surfaces. Place the unit in an area that is clean and free of dust. Refer to **page 2, Figure A**.

2. Check filter monthly or every 10-12 treatments. Details regarding when to replace the filter are covered under "Maintenance" on page 10.

3. Connect the polarized plug on the power cord to a polarized wall outlet. Refer to **page 2, Figure B**. Open the nebulizer package and read and follow the nebulizer's Instructions For Use.

5. Clean nebulizer per nebulizer Instructions For Use.

6. It is important to firmly push the Wing Tip™ Tubing all the way onto the air outlet of the compressor prior to starting the treatment. Refer to **page 2, Figure C**. The tubing is designed to fit tightly.

7. Turn compressor switch to ON position. Take treatment per your nebulizer Instructions For Use.

8. Press compressor switch to OFF position at the end of treatment.

9. Patients who use a 2.5 ml volume fill will generally find that the **PRONEB**® ULTRA II Compressor will deliver the treatment in less than 10 minutes. Treatment time is always a variable based on the patient's age and condition. The **PRONEB**® ULTRA II Compressor was designed for intermittent use to deliver respiratory medications, not continuous use for humidification or prolonged treatments (over 35 minutes at one time). It is possible that with prolonged treatments the tubing may get warm and disconnect from the **PRONEB**® ULTRA II Compressor. If you are a patient and will be using the **PRONEB**® ULTRA II Compressor for back to back treatments

that are longer than 35 minutes combined, please contact PARI Patient Connection at **1.800.FAST.NEB (327.8632), Press 2**.

*Note: Your **PRONEB**® ULTRA II Compressor has a nebulizer holder on the rear of the unit. Refer to **page 2, Figure C**. This nebulizer holder allows the nebulizer to be placed in the rear of the **PRONEB**® ULTRA II Compressor with the Wing Tip™ Tubing attached.*

## VI. Specifications

### Compressor Specifications:

Weight: \_\_\_\_\_ 3 lbs. or 1300 grams

Dimensions: \_\_\_\_\_ 6.5 x 6.5 x 3.75 in.

Electrical Requirements: \_\_\_\_\_ 120V AC, 60Hz, 1.5A

Power Wattage: \_\_\_\_\_ 80 W, under normal load

Ambient Operating  
Temperature Range: \_\_\_\_\_ 10°C to 40°C, (50°F to 104°F)

Power Cord Length: \_\_\_\_\_ 5 ft.

### Transport/Storage Range:

Ambient Temperature: \_\_\_\_\_ -40°C to 70°C, (-40°F to 158°F)

Relative Humidity: \_\_\_\_\_ 10% to 95%

Atmospheric Pressure: \_\_\_\_\_ 500 hpa to 1060 hpa

## Explanation Of Symbols:

OFF/ON: \_\_\_\_\_

Class II Equipment: \_\_\_\_\_

Type B Applied Part: \_\_\_\_\_

Ordinary Equipment (IPX0) According to the Ingress of Water.

Mode of Operation: \_\_\_\_\_ continuous

## The Following Tables Are Provided In Accordance With IEC 60601-1-2:

Guidance and manufacturer's declaration –electromagnetic emissions		
The PRONEB® ULTRA II is intended for use in the electromagnetic environment specified below. The customer or the user of the PRONEB® ULTRA II should assure that it is used in such an environment.		
Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The PRONEB® ULTRA II uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The PRONEB® ULTRA II is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Not applicable	
Voltage fluctuations / flicker emissions IEC 61000-3-3	Not applicable	

### Guidance and manufacturer's declaration –electromagnetic immunity

The PRONEB® ULTRA II is intended for use in the electromagnetic environment specified below. The customer or the user of the PRONEB® ULTRA II should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	±6 kV contact  ±8 kV air	±6 kV contact  ±8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	±2 kV for power supply lines  ±1 kV for input/output lines	±2 kV for power supply lines  ±1 kV for input/output lines	Mains power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1 kV differential mode  ±2 kV common mode	±1 kV differential mode  ±2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % $U_T$ (>95 % dip in $U_T$ ) for 0,5 cycle  40 % $U_T$ (60 % dip in $U_T$ ) for 5 cycles  70 % $U_T$ (30 % dip in $U_T$ ) for 25 cycles  <5 % $U_T$ (>95 % dip in $U_T$ ) for 5 sec	<5 % $U_T$ (>95 % dip in $U_T$ ) for 0,5 cycle  40 % $U_T$ (60 % dip in $U_T$ ) for 5 cycles  70 % $U_T$ (30 % dip in $U_T$ ) for 25 cycles  <5 % $U_T$ (>95 % dip in $U_T$ ) for 5 sec	Mains power quality should be that of a typical commercial or hospital environment. If the user of the PRONEB® ULTRA II requires continued operation during power mains interruptions, it is recommended that the PRONEB® ULTRA II be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A / m	3 A / m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE  $U_T$  is the a.c. mains voltage prior to application of the test level.

### Guidance and manufacturer's declaration –electromagnetic immunity

The PRONEB® ULTRA II is intended for use in the electromagnetic environment specified below. The customer or the user of the PRONEB® ULTRA II should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
<p>Conducted RF IEC 61000-4-6</p> <p>Radiated RF IEC 61000-4-3</p>	<p>3 Vrms 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2,5 GHz</p>	<p>3 Vrms 150 kHz to 80 MHz</p> <p>3 V/m 80 MHz to 2,5 GHz</p>	<p>Portable and mobile RF communications equipment should be used no closer to any part of the PRONEB® ULTRA II, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p><b>Recommended separation distance</b></p> $d = \left[ \frac{3,5}{V_1} \right] \sqrt{P}$ $d = \left[ \frac{3,5}{E_1} \right] \sqrt{P} \quad 80 \text{ MHz to } 800 \text{ MHz}$ $d = \left[ \frac{7}{E_1} \right] \sqrt{P} \quad 800 \text{ MHz to } 2,5 \text{ GHz}$ <p>where <math>P</math> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and <math>d</math> is the recommended separation distance in metres (m).</p> <p>Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, <sup>a</sup>should be less than the compliance level in each frequency range. <sup>b</sup></p> <p>Interference may occur in the vicinity of equipment marked with the following symbol:</p> 

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.

NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

<sup>a</sup> Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the PRONEB® ULTRA II is used exceeds the applicable RF compliance level above, the PRONEB® ULTRA II should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the PRONEB® ULTRA II.

<sup>b</sup> Over the frequency range 150 kHz to 80 MHz, field strengths should be less than  $[V_1]$  V/m.

Recommended separation distances between portable and mobile RF communications equipment and the PRONEB® ULTRA II			
The PRONEB® ULTRA II is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the PRONEB® ULTRA II can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the PRONEB® ULTRA II as recommended below, according to the maximum output power of the communications equipment.			
Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz to 80 MHz $d = \left[ \frac{3,5}{V_1} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[ \frac{3,5}{E_1} \right] \sqrt{P}$	800 MHz to 2,5 GHz $d = \left[ \frac{7}{E_1} \right] \sqrt{P}$
0,01	.12	.12	.23
0,1	.37	.37	.74
1	1.17	1.17	2.33
10	3.69	3.69	7.38
100	11.67	11.67	23.33
For transmitters rated at a maximum output power not listed above, the recommended separation distance <i>d</i> in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where <i>P</i> is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.			
NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.			
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.			

## VII. Maintenance Of The PRONEB® ULTRA II Compressor

### Wing Tip™ Tubing Maintenance:

If there is any moisture or condensation in the Wing Tip™ Tubing after the treatment, let the compressor run with only the tubing attached for a few minutes. The warm air from the compressor will dry the moisture or condensation in the tubing. Use a damp cloth to wipe the exterior of the tubing.

### Filter:

The Air Filter on the front panel of the PRONEB® ULTRA II Compressor is bright white. In time, the Air Filter will turn gray, which means it is time to replace it. You must replace the filter immediately if it gets wet or becomes clogged. The Air Filter will last for up to 500 hours before replacement. A good general rule is to replace the Air Filter whenever you replace your PARI Reusable Nebulizer (every 6 months). To remove the Air Filter, make sure the PRONEB® ULTRA II Compressor is turned off. Use a coin to turn the Air Filter cover counter clockwise. Pull the cap and filter out and remove the filter from the cap. Refer to **page 2, Figure D**. To replace the filter, press the new filter into the cap with the open end visible. Place the cap and filter back into the front of the compressor. Use the coin to turn the filter cover clockwise to tighten. Refer to **page 2, Figure E**.

*Note: For filter replacement obtain a new filter, part #041E4851P12. Do not attempt to wash and reuse the Air Filter. Do not use substitute filter material. Damage to the PRONEB® ULTRA II Compressor may result.*

#### **Cleaning the PRONEB® ULTRA II Housing and Wing Tip™ Tubing:**

Use a damp cloth to wipe the exterior of the PRONEB® ULTRA II Compressor housing and Wing Tip™ Tubing. Refer to **page 2, Figure F.**

Never spray fluid into the ventilation slots on the PRONEB® ULTRA II Compressor. Fluid could cause damage to the electrical and compressor parts and could lead to a malfunction. In the event that fluid enters the unit, return the product to an authorized PARI Service Center for examination and repair.

#### **Maintenance and Repair:**

Never operate the PRONEB® ULTRA II Compressor if it is improperly or incompletely assembled, or damaged. Use only original PARI parts and attachments authorized by PARI for the PRONEB® ULTRA II Compressor. Use of other parts and attachments may damage the unit or cause improper treatment. Do not open or disassemble the compressor unit at any time.

*Note: There are no serviceable parts inside the PRONEB® ULTRA II Compressor case. Service by anyone voids warranty.*

## **VIII. PRONEB® ULTRA II Compressor Limited Warranty**

PARI offers a standard five (5) year warranty on the PRONEB® ULTRA II Compressor and its parts. The warranty period will be printed on the serial number label located on the bottom of the compressor. PARI warrants that the PRONEB® ULTRA II Compressor and its parts will be free from defects in workmanship and materials for a period of sixty (60) months, from the date of first retail purchase of the equipment. This warranty does not cover failures of the nebulizer, tubing, or accessories. This warranty does not extend to failures resulting from accident, misuse, abuse, alteration, use of unauthorized service; parts or cleaning solutions, or failure to comply with Instructions For Use. If the warranted equipment should fail during the warranty period, PARI will replace the equipment or parts at issue, provided the claim is bona fide. It shall be the responsibility of the purchaser to pack and return the equipment in a manner to avoid shipping damage. The shipment must be prepaid to PARI Respiratory Equipment, Inc. This warranty covers the cost of replacement of the warranted items only. To make a claim under this warranty, contact PARI Patient Connection at:

**1.800.FAST.NEB (327.8632), Press 2**

THERE ARE NO OTHER EXPRESS WARRANTIES. ALL IMPLIED WARRANTIES ON THE PRONEB® ULTRA II COMPRESSOR AND ITS PARTS, INCLUDING IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE LIMITED TO SIXTY (60) MONTHS FROM THE DATE OF FIRST RETAIL PURCHASE. ALL EXPRESS OR IMPLIED WARRANTIES ON ANY OTHER PARTS, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, ARE DISCLAIMED. Some states do not allow limitations on how long an implied warranty lasts, so the above limitation may not apply you. The sole and exclusive remedy under this warranty is to replace the PRONEB® ULTRA II Compressor and/or its parts by PARI. IN NO EVENT SHALL PARI BE LIABLE FOR INCIDENTAL OR CONSEQUENTIAL DAMAGES TO THE PURCHASER OR

ANY THIRD PARTY, INCLUDING WITHOUT LIMITATION, DAMAGES ARISING FROM LOST PROFITS, LOSS OF BUSINESS OPPORTUNITY, LOSS OF PROPERTY, ECONOMIC LOSSES, OR STATUTORY OR EXEMPLARY DAMAGES, WHETHER IN NEGLIGENCE, WARRANTY, STRICT LIABILITY OR OTHERWISE. Some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific rights. You may also have other rights which vary from State to State.

## IX. Return Procedure

1. Contact PARI Patient Connection at **1.800.FAST.NEB (327.8632)**, Press 2 for a customer service representative.
2. In writing, describe the malfunction or symptoms of the **PRONEB® ULTRA II** Compressor. Include your full name, address and daytime telephone number.
3. Send written description and **PRONEB® ULTRA II** Compressor in a protective box to the address provided by Patient Connection.

Specialists in effective inhalation

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© 2008 PARI Respiratory Equipment, Inc.